USER MANUAL/GEBRAUCHSANLEITUNG MANUALE D'USO/MANUEL D'EMPLOI

(ENG - DEU - ITA - FRA)

Device / Vorrichtung/ Dispositivo / Appareil

KLICK HYBRID

Dear customer,

Thank you for choosing a KLICK product.

We sincerely hope that the KLAXON MOBILITY products can improve your day and your activities in many ways, and we remain at your disposal for any information or clarification you may require. We also encourage you to keep up to date on the latest news of our product range, on the accessories available and on our company's events and initiatives.

You can contact us by writing an email to info@klaxon-klick.com, through our web site at www.klaxon-klick.com, or on our Facebook page www.facebook.com/klaxonklickworld

Please note also that in compliance with European regulation, KLAXON MOBILITY GMBH holds CE certification for all the products in the KLICK range; they are classified as Class I medical devices and comply with the essential safety requirements set out in Annex I to Directive 93/42/EEC, as amended by the 2007/47/EC directive.

Thank you again for having chosen our products. We wish you a good day and enjoyable use.

Best regards,

The Staff of KLAXON MOBILITY GMBH

CONTENTS

PRELIMINARY INFORMATION	P. 3
1. WARNINGS	P. 3
2. SAFETY NOTICES	P. 4
3. INTENDED USE OF THE DEVICE	P. 4
4. USAGE ENVIRONMENT	P. 4
5. OPERATING LIMITS	P. 5
6. USER REQUIREMENTS	P. 5
7. WARNINGS AND PRECAUTIONS	P. 5
8. MARKINGS	P. 8
9. DESCRIPTION OF THE DEVICE	P. 8
9.1 PRODUCT STRUCTURE	P. 8
9.2 KLICK® CONNECTION	P. 8
9.3 BATTERY CHARGER	P. 9
9.4 ACCESSORIES	P. 9
9.5 DOCUMENTATION	P. 9
10. INSTALLATION AND COUPLING	P. 9
10.1 INSTALLATION	P. 9
10.2 COUPLING STEPS	P. 9
10.3 WHEELCHAIR	P. 10
11. KLICK HYBRID STANDARD VERSION OPERATION	P. 10
11.1 USER INTERFACE	P. 10
11.2 TURNING ON AND SETTING UP THE DEVICE	P. 10
11.3 USE	P. 11
11.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR	P. 12
12. CLEANING	P. 13
13. SERVICING	P. 13
13.1 PERIODICAL MAINTENANCE	P. 13
13.2 MALFUNCTIONS	P. 13
13.3 POWER BATTERY (11 AH)	P. 14
14. TECHNICAL DATA	P. 15
14.1 CLASSIFICATION OF THE DEVICE	P. 15
14.2 PERFORMANCE	P. 15
14.3 DIMENSIONAL DATA KLICK HYBRID STANDARD	P. 16
14.4 ENVIRONMENTAL OPERATING CONDITIONS	P. 16
14.5 TRANSPORTATION AND STORAGE ENVIRONMENTAL CONDITIONS	P. 16
14.6 TECHNICAL DESCRIPTION OF BATTERY CHARGER	P. 16
14.7 TECHNICAL DESCRIPTION OF BATTERY	P. 16
14.8 MAIN FEATURES	P. 16
15. DISPOSAL OF THE DEVICE	P. 16
16. PRODUCT WARRANTY	P. 17
17. ORIGINAL TEXT	P.17

PRELIMINARY INFORMATION

INTRODUCTION

This User's Manual explains how to use the KLICK Hybrid series devices that increase the independence of persons with disabilities.

MANUFACTURER'S IDENTIFICATION DATA

KLAXON MOBILITY GMBH

Industriestrasse 1
9601 - Arnoldstein (Austria)
Tel.: +43 (0)664 4681294
e-mail: info@klaxon-klick.com
Hereinafter: **KLAXON**.

REVISION INFORMATION

REV.	Description of the change	DATE
01	Initial release	30.06.2018
02	First revision	18.06.2020

All information, illustrations and specifications in this document are based on the latest product information available at the time of publication.

This document is subject to change due to technical innovations, without prior notice.

1. WARNING

- Read these instructions carefully before using the device.
- KLAXON declines all responsibility for failure to comply with the contents of this document.
- The use of KLICK devices is specific for persons with disabilities who use a manual wheelchair.
- The use of KLICK devices in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which they are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
- If any anomalies are found during use that may endanger the user, do not use the device and promptly contact KLAXON's technical support.
- Non-original spare parts and accessories have not been tested by the manufacturer. Therefore we cannot certify that these components are compliant with the needed performance and safety requirements.
- KLAXON is not responsible for any damage caused by the use of non-original spare parts or accessories.
- The product installation must be performed in accordance with the procedures defined by KLAXON, by authorised personnel. Neither the safety nor the effectiveness of the device can be guaranteed if these procedures are not followed, nor can the validity of the warranty.

The points of the manual marked with the following symbols should be read carefully:

DANGER	The word DANGER indicates a potentially hazardous situation, which, if not avoided, can result in death or serious personal injury.
CAUTION	The word CAUTION indicates a potentially hazardous situation, which, if not avoided, may cause minor personal injuries or property damage.
NOTE	The word NOTE indicates important instructions or special information.
	Generic warning sign: indicates a danger due to various causes.
	High voltage warning sign: indicates the presence of high voltages.

2. SAFETY NOTICES

	The user must read this document and have become familiar with its contents before using the device.
	If the device is not used respecting the specifications described herein, the security level specified by the manufacturer may be reduced.
	During the initial period of use the user must familiarize with operation to avoid potential risks caused by incorrect use.
	EMC - The device is compliant with ISO 7176-21:2009 CLAUSE 5.4 on electromagnetic compatibility, but to ensure the user's safety we recommend not using this device in the presence of strong environmental electromagnetic fields.

Children, incapacitated persons, or those that are insufficiently familiar with the device or physically unsuited, must not use it.

- The device may skid on wet pavement, on gravel or on uneven terrain: always adjust speed and driving style to the conditions (weather, surface, individual capacity, etc.); never drive straight towards an object, and drive carefully towards narrow passages.
- Do not attempt to go over steps higher than 50 mm, both uphill and downhill, along the direction travelled. Do not drive the KLICK-equipped wheelchair on roads with slopes that exceed 6°. These obstacles must always be driven over head-on and slowly. The proper way to go over steps and to go up slopes is shown in Figure 1.
- If the pavement is wet or uneven, the maximum slope and the maximum height of the steps allowed must be considered lower to ensure the user's safety.
- In the event of a collision, the user could suffer serious injury to body parts.
- At high speed the user may lose control of the wheelchair and it may tip over. Never exceed 25 km/h, and avoid impacts in general.
- Always use the KLICK-equipped wheelchair suitably, always braking to avoid obstacles (steps, edges, door frames, etc.) or to descend from curbs or go down slopes.
- Driving performance of the KLICK-equipped wheelchair may be affected by electromagnetic fields. If a malfunction occurs in these conditions, please contact KLAXON or one of its authorised resellers.
- Avoid contact with any liquid coming out of the battery if it is damaged.
- While using the device, the brake system components can get very hot and contact of these components with skin may cause scalding and burns: be extremely careful to avoid contact.
- Always observe the operating limits for the manual wheelchair on which the KLICK system is installed.

3. INTENDED USE OF THE DEVICE

KLICK devices can be used to add an auxiliary power assist system to most manual wheelchairs, quickly and easily.

Coupling the KLICK device to the wheelchair, the front castors rise off the ground and a single wheel, in the "traction unit", is used for steering.

KLICK devices are classified for use with wheelchairs to which they are coupled pursuant to CEI EN 12184. KLICK devices are also compliant with the requirements of class B with maximum range of about 50 km in "electric" version (on a flat course and 75 kg user) and maximum range of about 100 km in "hybrid" version (on a flat course and 75 kg user).

In accordance with the definition of class B of CEI EN 12184, the device is for indoor use and can handle outdoor obstacles.

	Any other use or misuse could result in hazardous situations.
---	---

4. USAGE ENVIRONMENT

Operating Environment

KLICK series devices are made for use coupled to most manual wheelchairs on the market. KLICK solutions can be used both indoors and outdoors, with the limits stated in this document, and in general with the same limitations set for the use of the manual wheelchair.

Medical use and types of users

KLICK is not intended for specific clinical use, but as a support to the mobility of persons who use manual wheelchairs. Therefore, bearing in mind the operating precautions described in this document, there is no need for professional, technical or aptitude requirements to operate or use a KLICK device.

5. OPERATING LIMITS

The device must be used consistently with the intended use described in the preceding paragraphs, any other must be considered improper use.

In particular, do not use KCLICK devices:

- If not coupled with a manual wheelchair.
- After intake of alcohol or if the user's psycho-physical conditions are unsuitable.
- On roads with unsuitable pavement.
- In poor visibility (at night, in fog, etc.).
- If the user weighs over 120 kg or is heavier than allowed for the wheelchair used.
- In adverse weather (rain, strong wind, etc.).
- When going uphill or downhill on slopes that exceed 6° (see Figure 2).
- To go over steps higher than 50 mm, both uphill and downhill (see Figure 3).

6. USER REQUIREMENTS

KCLICK Hybrid series devices and its accessories can be used independently by users that:

- That are familiar and have practised proper use of the device sufficiently.
- That are aware of the risks related to the use of the device.
- That have sufficient physical and mental capacity to ensure the appropriate safe use of the device

7. WARNINGS AND PRECAUTIONS

	This User's Manual is an integral part of the device and must always be available to the user. If it is lost or damaged, contact KLAXON.
	KCLICK series devices and accessories can be used by users who are aware of the risks associated with the use thereof.
	KCLICK series devices and accessories must be used only and exclusively by users that are familiar and have practised proper use of the device sufficiently.
	Absolutely never remove, modify or replace components of the device. If necessary, these operations must be carried out only by technical personnel authorised to do so by KLAXON.
	Never use the device after having drunk alcohol, taken drugs and in general if the user's psycho-physical conditions are unsuitable.
	Do not use KCLICK devices for users weighing more than 120 kg; in total, the weight must not exceed the amount specified by the manufacturer of the wheelchair used.
	The user is responsible for monitoring and maintaining proper operation of the devices and of their accessories as indicated herein.
	Absolutely never use the device outside of the stated environmental operating conditions.
	Periodic maintenance must be performed to ensure proper operation and the safety of the device.
	Do not use a KCLICK device to carry more than one person.
	Do not use a KCLICK device unless it is coupled with a manual wheelchair.

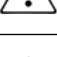
KLAXON Mobility GmbH - ENG

	Do not use the supplied battery pack to power systems other than the KLICK device for which it was designed.
	Do not leave the battery in the sun or adverse weather when not in use.
	Do not short the battery.
	Do not throw the battery in water.
	Do not attempt to open the battery pack.
	Do not allow children to play with the battery.
	If the battery has reached its end of life, it must be disposed of in the special containers at designated collection centres.
	While using the device, the brake system components can get very hot and contact of these components with skin may cause scalding and burns: be extremely careful to avoid contact.

	DANGER: The voltages inside the device can be lethal.
	DANGER: Do not connect/disconnect power cables when the device is on.
	DANGER: Maintenance or cleaning the device must always be carried out when the device is off and disconnected from the mains.
	DANGER: Do not remove any of the unit's cover panel, contact the Technical Support Service.


	The use of KLICK devices in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which they are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
	Adjust speed in relation to the characteristics of the wheelchair, the road conditions, the lateral inclination of the road and the radius of the curves to take.
	Do not operate the device in poor lighting conditions. The device is not suitable for use at night.
	Pay attention to sideways listing due to strong wind.
	Pay attention to sideways listing due to steering while moving forward.

KLAXON Mobility GmbH - ENG

	Pay attention to sideways listing due to the slope of the road.
	Do not use KLICK devices on roads with unsuitable pavement.
	Do not use KLICK devices with adverse weather (rain, strong wind, etc.).
	Do not use KLICK devices to go up or down steps higher than 50 mm.
	Do not use KLICK devices on slopes greater than 6°
	When driving downhill, do not exceed two-thirds of the maximum speed; avoid sudden braking or accelerations on slopes.
	Never go over obstacles when driving uphill or downhill.
	Do not go up or down staircases.
	Do not attempt to go over obstacles diagonally: make sure that the rear wheels always go over the obstacle together. Never stop halfway. Do not attempt to go over obstacles over 50 mm high.
	When the vehicle is in motion, avoid moving the centre of gravity or abruptly moving your body.
	It is advisable to drive through narrow passages at minimum speed and with utmost caution.
	When using KLICK devices, the risk of tipping over during cornering increases: reduce speed before turning. Accelerate again only after completing the curve.
	When using KLICK devices, do not make U-turns on slopes greater than 3° and at a speed higher than level 1.
	It can be difficult to manoeuvre in front of a lift or at the entrance of a building since the wheelchair's turning radius increases when coupled with a KLICK device; this could conflict with the standards of the buildings or the actual size of the entrances.
	When entering a building and/or an elevator, always take into account the turning radius of your wheelchair equipped with the KLICK device. Avoid driving into situations from which you would not be capable of exiting because you would be unable to turn your wheelchair.
	Absolutely never use the Klick device along tramway, rail and light railway tracks. Pay the utmost attention at level crossings, keeping the Klick's and wheelchair wheels at due distance. Failure to comply with this precaution could cause serious damages to the device and the user. The same precautions apply to water drains, manholes and all kinds of hollows along the way.

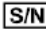8. MARKINGS

The device is uniquely identified by its serial number (SN), shown on the plate of the device, attached to the motor unit near the battery. The meaning of the symbols shown in the plate are as follows:

Symbol	Description
	Manufacturer
	Reference number
	Serial number
	Year of manufacture
	Symbol indicating: Follow the instructions for use
	Part B applied part
	CE mark
	Electrical device disposal via separated collection (special disposal, electrical components)

Pictures of the CE Label, applied on the Klick devices.

9. DESCRIPTION OF THE DEVICE

9.1 PRODUCT STRUCTURE

Figure 4 shows the KLICK HYBRID STANDARD device and identifies the elements that make it up.

1. Steering wheel
2. Brake system
3. Motor unit
4. Kickstand
5. 9-speed manual transmission
6. Battery
7. Left hand grip equipped with the brake lever
8. Right hand grip equipped with the brake lever and the manual gear lever
9. Display, accelerator lever and control pushbutton panel

9.2 KLICK® CONNECTION

See Figure 5.

A. CLAMPING RINGS

The clamping rings are the parts of the KLICK connection that always remain attached to the wheelchair frame (see Figure 6). Together with the crossbeam, they must be installed on the wheelchair by authorised KLAXON personnel and must never be removed by the user during use. Various clamping ring sizes and shapes are available to suit the different tubing diameters and types of wheelchair frames.

B. CROSSBEAM

Figure 7 illustrates a CROSSBEAM and identifies the elements that make it up.

1. CROSSBEAM
2. Square seats for insertion of the CONNECTOR
3. Fixing pins
4. Fastening and adjustment straps.

C. CONNECTOR (WITH DETAIL OF PATENTED "KLICK® SYSTEM")

Figure 8 illustrates a STANDARD CONNECTOR and identifies the elements that make it up.

1. CONNECTOR lock/unlock lever
2. Mounting system in the CROSSBEAM (with ball-type lock)
3. KLICK device release button
4. KLICK® coupling (Worldwide Patented)
5. Seat of upper pin
6. Seat of lower pin.

D. TETRA CONNECTOR (WITH DETAIL OF PATENTED "KLICK® SYSTEM")

Figure 9 illustrates a TETRA CONNECTOR and identifies the elements that make it up.

1. Velcro grip of CONNECTOR
2. Mounting system in the CROSSBEAM (with ball-type lock)
3. Easy release lever of the KLICK device
4. KLICK® coupling (Worldwide Patented)
5. Seat of upper pin
6. Seat of lower pin.

9.3 BATTERY CHARGER

Figure 10 shows the battery charger that is supplied with the POWER battery (11 Ah), which are compatible with the Hybrid model.

9.4 ACCESSORIES

KLICK can be equipped with various accessories. You may need to consult the appropriate User's Manual (ask KLAXON or visit our website: www.klaxon-klick.com) for some of them.

9.5 DOCUMENTATION

This USER'S MANUAL is provided with the Device in printed format and must be stored with the utmost care. If you lose the above documents, request a copy of them in writing to KLAXON.

10. INSTALLATION AND COUPLING

10.1 INSTALLATION

KLICK devices are provided by KLAXON (or by an authorised technician) only after having pre-installed the coupling set on the wheelchair.

10.2 COUPLING STEPS

During the entire coupling operation, stop in a flat area, as free as possible from uneven ground and away from hazards which could cause personal injury.

Insert the CROSSBEAM in one of the two CLAMPING RINGS and secure it by inserting the specific PIN (Figure 11).

- Insert the CROSSBEAM also in the second CLAMPING RING and secure it by inserting the specific PIN (Figure 12).
Note: The CROSSBEAM can always remain installed on the wheelchair; if necessary (for loading and unloading from vehicles and closing the folding wheelchair) it can be removed and placed in a safe place.
- Insert the CONNECTOR in the square seat of the CROSSBEAM, pressing the lever located on its bottom. Once inserted, release the lever and make sure that the CONNECTOR is securely locked (Figure 23).
- After having inserted the CONNECTOR, the device will be as shown (Figure 14).
- To secure the KLICK device to the connection, move the wheelchair into position as shown (Figures 15 and 16). Make sure the opening of the coupling is placed over the upper pin (Figure 17).

- Maintaining the position, activate the brakes of the wheelchair and grip the pedal cranks at the centre, near the point of insertion with the movement of the steering wheel; lift the KLICK device until the upper pin is fully inserted in the opening of the coupling (Figure 18).
- At this point, continue to raise the handlebar by pushing it forward until the lower pin is fully engaged with the connection; at that point the safety button is triggered (Figure 19).

CAUTION!

Once the device is connected to the wheelchair, it is important to ensure that:

- The ground clearance of the front wheels of the wheelchair is 40 mm (with the wheelchair empty - Figure 20).

The kickstand is located above the line of the front wheels and placed below the footboard, so as not to interfere with it (Figure 21).

CAUTION	If these requirements are not met, never use the device for any reason and immediately contact an authorised KLAXON service centre.
---------	---

10.3 WHEELCHAIR

KLICK devices can be installed only on compatible wheelchairs. KLICK products must be only be installed by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with the KLICK device.

11. KLICK HYBRID STANDARD VERSION OPERATION

11.1 USER INTERFACE

The device is equipped with a multifunction LCD display, used to switch the device on and off, select the maximum speed and display several operating parameters (Figure 22).

- The icon located top left (1) shows the current battery charge level.
- The number (2) in the centre of the display shows the speed, in km/h.
- The number (3) in the lower part of the display shows the distance travelled, in km.
- The number (4) represents the desired assistance level (in hybrid and electric mode).
- The number (5) indicates the USB socket used to recharge devices (smartphone, front LED, etc.).

11.2 TURNING ON AND SETTING UP THE DEVICE (FIGURE 22 E 23).

A. Turning on and selecting the speed

- Press and hold down the **"POWER"** key for 5 seconds, until the display comes on.
- Press the **"+"** and **"-"** keys to select the desired maximum assistance level (shown in the box on the left side of the display, number (4) in the legend); the electric power assistance levels including, for example, 0 (motor deactivated), 1 (minimum level, 10%), 2 (normal level, 20%), 3 (medium level, 40%), 4 (high level 60%), 5 (maximum level 80%) can be selected.

B. Data navigation

- When the display comes on, the initial screen shows (in number 3) the partial distance (TRIP) of use of the device.
- If you press the **"INFO"** key, the total distance is shown (TRIP – number 3).
- If you press the **"INFO"** key again, the maximum speed reached will be shown (MAXS – number 3).
- If you press the **"INFO"** key another time, the average speed will be shown (AVG – number 3).
- Pressing the **"INFO"** key will show the residual charge of the battery, expressed in km, according to the assistance level selected (TO GO – number 3).
- Pressing the **"INFO"** key again will show the instantaneous power absorption of the motor expressed in watts (number 3).

C. Data reset

- When the display is turned on, press the **"INFO"** key twice in quick succession to enter the MENU. At this point, the submenu TC will appear (number 2).
- Press the **"+"** key to switch from **"N"** to **"Y"** (number 3); at this point, the partial distance and speed data will be reset.
- Press the **"INFO"** key twice to exit from the MENU and return to the initial screen..

D. Screen backlight

- If you press and hold down the **"LIGHT"** key for 5 seconds, the screen backlight will be turned on.
- To turn off the backlight, press the **"LIGHT"** key again for 5 seconds.

E. Maintain speed function

- Press and hold down the “-” key for 5 seconds to activate the Maintain speed function automatically. This function increases the speed of the device to 4.5 km/h and maintains it for as long as the pushbutton is held down.
- When the key is released, the device will stop accelerating instantaneously.

DANGER	Every action and/or tampering of the display to change performance and KLICK’s security is strictly prohibited. The sole and only actions possible are those specified in paragraph 11.2 of this User’s Manual.
---------------	---

NOTE: For further information on the display, see www.klaxon-klick.com or contact an authorised KLAXON dealer.

11.3 USE

Figures 24 and 25 illustrate the pedalboards of the Klick device and identify the elements that make them up (listed here below).

1. Cranks
2. Accelerator control
3. Display control button panel
4. Display
5. Brake grip and lever
6. Manual transmission controls

The device has no reverse gear. To move in the opposite direction, turn the wheelchair using the push-rims on the wheels of the wheelchair; while doing this, keep one hand on the grips of the device. This will enable you to control the direction of movement and prevent the cables (electrical and mechanical) from getting twisted with consequent damage and/or breakage.

CAUTION	Using the Klick device in reverse gear could cause it to malfunction if the cables get damaged and/or broken and accidentally activate the accelerator lever if the device is on with a forward gear engaged. With regard to this, before moving backwards, the forward gear must be set to “0” and the device must be turned off. Failure to follow these indications could cause damage to the user and third parties (people and/or property).
----------------	---

A. ELECTRIC MODE

Having installed the device, turned on the display and selected a forward speed, the device will start to move when the accelerator control is activated (Figure 43). To stop the device, activate the brake lever (Figure 26). The KLICK brake is equipped with an additional motor deactivating safety mechanism when the lever is activated.

DANGER	If there are any obstacles, drops or hazards of any other kind not situated at a safe distance, turn off the KLICK device and move the wheelchair to a safe place.
DANGER	When using the Klick device in electric mode it is mandatory to keep one hand on the handle so that the cranks cannot rotate. Failure to do so may result in the cables twisting and breaking. It is absolutely forbidden to use the accelerator lever and the cranks at the same time.

B. HYBRID MODE

Having installed the device and turned on the display, when the cranks are turned forwards, the wheelchair will begin to move (Figure 27). According to the forward speed selected (which will determine the percentage of electric power assistance – see Paragraph 11.2), the motor will sense the forward movement of the cranks (with a delay of half a revolution with respect to their rest position) and will be activated accordingly, moving the wheel with electric power assistance.

Whenever the cranks stop moving or are turned in the opposite direction (backwards), the motor will be deactivated automatically after 1 second.

Figure 28 illustrates the operation of the manual transmission used to change the ratio with the wheel; this modifies the gear ratio in a range from 1 (the highest – for moving uphill and starting after a stop) to 9 (the lowest – suitable for flat and downhill sections). The ratio selected at that time can be seen through a mechanical indicator set on the top of the gearbox. Its operation is determined as follows:

- Pushing the lever with your thumb (1), the ratio will increase (from 1 to 9)
- Pulling the lever (2) towards you with the index finger, the ratio will decrease (from 9 to 1).

The selection of the manual gear ratio does not change the degree of assistance of the electric motor; the two systems coexist autonomously and it is the user who decides at his own discretion (on the basis of his capabilities, the route to be followed, etc.) which level to select for both. We suggest that high ratios (over number 4) should never be used to start moving, in that the force applied to the cranks could tension the chain excessively and consequently the toothed wheels present on the wheel and handlebar.

To stop the device, activate the brake lever (Figure 26); The KLICK brake has an additional safety mechanism to deactivate the motor when the lever is activated.

CAUTION	Before use, check that the chain is correctly tensioned; move your finger to a point half way between the crown and the return on the selector fork and press it to the right and to the left. These oscillations must be less than 20 mm wide.
DANGER	For obstacles, drops and all other kinds of hazards not situated at a safe distance, turn off the KLICK device and move the wheelchair to a safe place.
DANGER	When using the Klick device in hybrid mode it is absolutely forbidden to use the accelerator lever and the cranks at the same time. It is necessary to keep both hands on the handles in order to avoid that the cranks, in absence of control, can rotate on themselves causing the cables to twist and consequently break.

C. MANUAL MODE

Having installed the device, keeping the display turned off or selecting the speed "0", it will start to move when the cranks are turned forwards (Figure 27). Figure 28 illustrates the operation of the manual transmission used to change the ratio with the wheel; this modifies the gear ratio in a range from 1 (the lowest – for moving uphill and starting after a stop) to 9 (the highest – suitable for flat and downhill sections). The ratio selected at that time can be seen through a mechanical indicator set on the top of the gearbox. Its operation is determined as follows:

- Pushing the lever with your thumb (1), the ratio will increase (from 1 to 9)
- Pulling the lever (2) towards you with your index finger, the ratio will decrease (from 9 to 1).

The selection of the manual gear ratio does not change the degree of assistance of the electric motor; the two systems coexist autonomously and it is the user who decides at his own discretion (on the basis of his capabilities, the route to be followed, etc.) which level to select for both. We suggest that high ratios (over number 4) should never be used to start moving, in that the force applied to the cranks could tension the chain excessively and consequently the toothed wheels present on the wheel and handlebar.

To stop the device, pull the brake lever (Figure 26); The KLICK brake has an additional safety mechanism to deactivate the motor when the lever is activated.

CAUTION	Before use, check that the chain is correctly tensioned; move your finger to a point half way between the crown and the return on the selector fork and press it to the right and to the left. These oscillations must be less than 20 mm wide.
DANGER	For obstacles, drops and all other kinds of hazardous not situated at a safe distance, turn off the KLICK device and move the wheelchair to a safe place.

11.4 TURNING OFF AND RELEASING FROM THE WHEELCHAIR

To do this correctly, carefully follow the following instructions .

- Stop the wheelchair on a flat surface.
- Activate the brakes of the wheelchair.
- Turn off the display, by pressing the **"POWER"** key and holding it down for 5 seconds.

A. STANDARD CONNECTOR

- Grip the centre of the handlebar, push it forwards and press the release button on the STANDARD CONNECTOR with your other hand at the same time. This will release the lower pin of the KLICK coupling (Figure 29).
- Reduce the pressure applied to the handlebar while holding the release button on the CONNECTOR pressed, so as to lower the wheelchair and bring the front wheels back down onto the ground (Figure 30).
- At this point, the upper pin will have already been released from the front hook of the CONNECTOR; release the brakes of the wheelchair and push it backwards so as to move it away from the KLICK device (which remains upright on its KICKSTAND - Figure 31).
- Remove the CONNECTOR by pressing the lever at the bottom of it (Figure 13).

B. TETRA CONNECTOR

- Push the quick release lever on the TETRA CONNECTOR firmly to the right. In this way, the lower pin will be released automatically and the KLICK device will be released from the CONNECTOR, lowering the wheelchair and bringing the front wheels back down onto the ground (Figures 32 and 33).
- At this point, the upper pin will have already been released from the front hook of the CONNECTOR; release the brakes of the wheelchair and push it backwards so as to move it away from the KLICK device (which remains upright on its KICKSTAND - Figure 31).
- Remove the CONNECTOR by pulling the Velcro grip upwards and forwards (Figure 42).

DANGER	When releasing the KLICK device, take the utmost care to avoid injury to the upper limbs (arms, hands and fingers) as they could be struck by the handlebar (or other parts of the device) moving suddenly if it is released too quickly. To avoid this risk, having pressed the release button, release the handlebar slowly so as to lower the wheelchair gently.
--------	---

DANGER	When releasing the Klick device with the Tetra CONNECTOR, the wheelchair may drop suddenly until the front wheels touch the ground. Take the utmost care to avoid injury to the upper limbs (arms, hands and fingers) as they could be struck by the handlebar or other parts of the device moving suddenly..
---------------	---

12. CLEANING

Regularly cleaning the entire device makes it last longer and work better.

We recommend:

- To switch the device off and to disconnect the battery charger before cleaning it.
- To not use abrasive cleaning agents, detergents and high pressure cleaners.
- To clean the device with a soft, damp cloth.
- To dry the unit thoroughly after using it out in the rain.
- In case the device is dirty, to soften the dirt and to remove it as soon as possible; then to dry the device carefully.
- To clean all the outside of the device at least once a month and after each ride on muddy terrains.
- To be careful not to wet the electrical parts (battery, display, motor).
- To not wash the parts under running water
- To dry the parts immediately after washing by using a soft cloth.
- If the electrical parts get wet, let them dry and do not use the device until they are completely dry.
- After cleaning the device, lubricate the chain and the manual transmission.

13. SERVICING

KLICK devices must be serviced regularly, to ensure safe, reliable and efficient operation.

KLAXON is not responsible for any damage caused by non-maintenance or by insufficient maintenance, and in said cases the warranty may lapse.

Unscheduled maintenance can be carried out only by technicians authorised by KLAXON's Technical Support Service.

13.1 PERIODICAL MAINTENANCE

We recommend the following regular maintenance:

- At least once a month, check the tyre inflation pressure; a flat tyre affects the vehicle's smooth ride, its safety and its power consumption.
- At least once a month, check wear of the tyres. To replace an inner tube or a tyre, contact an authorised KLAXON technician.
- At least once every two months, check the tightness of all screws and fasteners.
- At least once a month, check the tightness of the fixing nuts of the wheels' pins with a torque wrench calibration at 50Nm.
- Before each use, check proper operation of the accelerator and brakes: any replacement of parts subject to wear (brake pads, discs, brake system in general) should be carried out only by authorised KLAXON technicians. The vehicle should also be checked whenever you feel the braking power has decreased.
- Check the condition of the battery, before each use. To replace the battery, and any other electrical component, contact only authorised KLAXON technicians.
- Before use, check the lubrication of the chain and the manual transmission.
- Before use, check that the manual transmission and all its drive elements are functioning correctly.

	Have a KLAXON qualified technician overhaul the entire device at least once a year, and/or whenever a malfunction occurs.
---	---

13.2 MALFUNCTIONS

If you encounter a malfunction during use of the KLICK device, contact KLAXON or of its authorised resellers.

Malfunction found	Possible cause	Corrective action
The device does not turn on	The battery is empty	Recharge the battery
	The battery is positioned improperly	Check the position of the battery by removing it and inserting it again, making sure that the electrodes are inserted in the appropriate spaces.
	General problems	Contact an authorised KLAXON service centre

KLAXON Mobility GmbH - ENG

It vibrates excessively during use	Loose screws	Contact an authorised KLAXON service centre
The display lights up but the device does not move	Disconnected plug	Contact an authorised KLAXON service centre
You have problems coupling the traction unit to the connector	The connector is not adjusted properly	Contact an authorised KLAXON service centre
You have problems attaching the connector	There is dirt inside the coupling seat in the crossbeam	Clean the surfaces of the seat in the crossbeam and of the mounting system in the connector
You have problems attaching the connector	The interlocking balls do not retract when pulling the lever	Contact an authorised KLAXON service centre
Wheelchair front wheels touch the ground	The Klick coupling is mounted too low	Contact an authorised KLAXON service centre
KLICK is noisy when in use	The fender touches the wheel while moving	Make sure your feet are in a position that does not interfere with the fender of the device Reposition the fender, putting it back into place.
	The brake pads touch the disc when reversing	Contact an authorised KLAXON service centre

13.3 POWER BATTERY (11 AH)

The POWER battery (48V and 11 Ah) is box-shaped and secured to the connection with a latching system (Figure 36).

On the top of the battery, there are markings along which some LEDs show the current charge level. Pressing the button, the LEDs will show the approximate charge level, in four levels (20%-50%-80%-100%) and will automatically activate the battery, powering the motor and the display (Figure 37). Use the same button to disable the battery (therefore the LEDs turn off).

The key, placed beside the battery, can be used to remove the entire unit from the device; by turning it, the battery is released and it can be removed by pulling it forward and upward (Figures 38 and 39).

To charge the battery:

- Turn the device off (by pressing and holding the **"POWER"** button);
- Remove the battery by turning the key (Figures 38 and 39).
- Remove the protection cap located on top of the battery (Figure 40).
- Connect the battery charger's plug (Figure 41);
- Connect the battery charger's plug to the 230V 50 Hz mains (Figure 34);
- The two LED lights on the upper surface of the battery charger (Figure 35) show its status:
 - Red-red: charging;
 - Green-red: charging complete, fully charged battery.
- When the battery is fully charged, first disconnect the plug from the mains socket and only then remove the plug from the battery.

NOTES:

- The batteries supplied with the Klick device are compliant with the EMC2004/108/EC regulation issued by the European Union regarding the CE Classification. It is therefore permitted to transport these batteries on a train, ship or airplane. Despite of this classification they are considered, according to the UN Recommendations on the Transport of Dangerous Goods, as dangerous material and classified as Class 9, respectively as: code UN3480 (Lithium Ion Batteries) if transported as single units and UN3481 (Lithium Ion Batteries Contained in Equipment) if transported while connected to the Klick device. This classification determines some restrictions regarding the transportation of the product by ship (IMDG Code) and by plane (IATA DGR) referring to: quantity of batteries transported, type of transport (if transported as single unit or connected to the device) and the Watt-hour value (Wh) of each single battery. In this regard, if you are planning to take your device with you, please contact directly the shipping company or the airline chosen for your travel.

- POWER battery: 48V 11Ah (528Wh)

- We recommend paying attention to the charge level indicator! Always recharge when the level is low. We always advise you to recharge the battery after extended use or overnight. Lithium-ion batteries have no memory effect, so they can be charged without having to wait until they are completely discharged.
- Store the battery charger away from heat sources, in a dry place and protected from direct sunlight.

KLAXON Mobility GmbH - ENG

	When not using the battery, remember to charge it at least once a month.
	If you see that the battery is damaged, its housing is broken, if it swells or leaks, absolutely do not use it and contact KLAXON's Technical Support immediately.
	Only use the supplied battery charger together with the KLICK device: any damage or malfunction due to non-compliance with these instructions or use of products that do not comply with KLAXON's guidelines will not be covered by the warranty.
	There is a risk of injury due to short-circuits and risk of electric shock if the battery charger has been damaged. Avoid using the battery charger if it has been dropped or damaged.
	Avoid using extension cords unless it is absolutely necessary. If you must use them, make sure in advance that they are undamaged and in excellent condition, to prevent fire hazards and electrical shock.
	Never recharge the batteries and operate the device at the same time, due to risk of injury to yourself and to others.
	Don't leave the battery charger connected to the 230 V mains for too long after it has finished its charge cycle (green LED on).
	Always charge the battery in a well ventilated room and keep it away from flammable material.

14. TECHNICAL DATA

14.1 CLASSIFICATIONS OF THE DEVICE

According to the Medical Devices Directive	Risk class I
Protection against electrical hazards	Class II
Degree of protection against direct and indirect contact	Type B
Use in oxygen-rich environments	No protection
Operating conditions	Device for continuous operation

14.2 PERFORMANCE

#	Feature	Value	Unit	Notes
1	Electric power assistance level	10; 20; 40; 60; 80	%	Electric and Hybrid modes
2a	Electric mode autonomy	45-50	km	With battery charged, moderate speed, flat route and a body weight of 75 kg
2b	Hybrid mode autonomy	90-100	km	With battery charged, moderate speed, flat route and a body weight of 75 kg
2c	Manual mode autonomy	∞	km	
3	Maximum gradeability	6	°	With battery charged and body weight of 75 kg
4	Maximum height of obstacle that the wheelchair can go over	50	mm	With battery charged and body weight of 75 kg
5a	Motor power	500	W	
5b	Electric motor voltage	48	V	
6	Maximum weight of user	120	kg	Consistently with the technical characteristics of the wheelchair to which it is attached.

KLAXON Mobility GmbH - ENG

14.3 DIMENSIONAL DATA KLICK HYBRID STANDARD

#	Feature	Value	Unit	Notes
1	Length	600	mm	
2	Width	500	mm	
3	Height	1250	mm	
4	Weight	18.0	kg	Weight only of drive unit without battery
5a	Weight of POWER battery (11 Ah)	3.0	kg	

14.4 ENVIRONMENTAL OPERATING CONDITIONS

#	Feature	Value	Unit	Notes
1	Maximum altitude	2000	m	
2	Atmospheric pressure	700 to 1060	hPa	
3	Temperature	-25 to 50	°C	0°C to 40°C is suggested for a better performance
4	Relative humidity	30 to 90	%	

14.5 TRANSPORTATION AND STORAGE ENVIRONMENTAL CONDITIONS

#	Feature	Value	Unit	Notes
1	Temperature	-40 to 65	°C	10°C to 50°C is suggested
2	Relative humidity	30 to 90	%	

14.6 TECHNICAL DESCRIPTION OF BATTERY CHARGER

#	Feature	Value	Unit	Notes
1	Input voltage	230	V	
2	Frequency	50	Hz	
3	Output Voltage	48	V	

14.7 TECHNICAL DESCRIPTION OF BATTERY

#	Feature	Value	Unit	Notes
1	Rated voltage	48	V	
2	Capacity	11	Ah	
3	Charging time	5.5	h	Time to charge a completely discharged battery

14.8 MAIN FEATURES

#	Feature	Value
1	Traction type	Electric
2	Brake	Disc (diameter 160 mm), mechanical
3	Type of use	Outdoor and indoor

15. DISPOSAL OF THE DEVICE

	<p>Disposal of the equipment inside the KLICK devices must absolutely comply with regulations of the country in which the devices are used. KLAXON is not liable for use that is not in accordance with specific regulations in force.</p>
---	--

16. PRODUCT WARRANTY

The warranty period is 24 months for the traction device and 12 months for the batteries, starting from the delivery date of the device.

Natural wear and tear of parts is not covered by the warranty, except for improper wear due to a manufacturing defect.

If a defect occurs during the warranty period, KLAXON will, at its discretion, repair or replace the defective component.

The manufacturer assumes no liability for damages caused by negligence, careless use and maintenance, tampering or incorrect maintenance or by unauthorised persons, or for failure to follow the instructions for use set out in this Manual.

17. ORIGINAL TEXT

This document is a translation of the original version written in Italian. Please note that in case of discrepancies between the two versions, the Italian version will be the one considered as official and valid.

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KLAXON MOBILITY GmbH