USER MANUAL - GEBRAUCHSANLEITUNG MANUALE D'USO - MANUEL D'EMPLOI

(EN - DE -IT- FR)

Device / Gerät / Dispositivo / Dispositif

KLAXON TWIST - FTD_30_WHEELCHAIR_FW-V3.1.0

Dear customer,

Thank you for choosing a KLAXON product.

We sincerely hope that KLAXON MOBILITY's products can improve your day and your activities in many ways and we remain at your disposal for any information or clarification you may require. We also encourage you to keep up to date on the latest news of our product range, on the accessories available and on our company's events and initiatives.

You can contact us by writing an e-mail to info@klaxon-klick.com, through our web site at www.klaxon-klick.com, on our LinkedIn page www.linkedin.com/company/klaxon-mobility or on our Facebook page www.facebook.com/klaxonklickworld.

Please note also that in compliance with European regulations, KLAXON MOBILITY GMBH holds CE Certification for all products in the KLAXON range; they are classified as Class I medical devices according to Annex VIII EU745/2017 MDR.

The TWIST devices belongs to:

- code GMDN 42805 (Wheelchair electric-motor-driven propulsion system)
- code UMDNS 17952 (Power Conversion Kits, Wheelchair)

	EU WARNING: In the event of any serious accident related to the TWIST device, the user must immediately notify the manufacturer and the competent authority of the European country where the user has bought the product.
NOTE	A PDF copy of this manual is available to people with visual impairment on our website www.klaxon-klick.com .
NOTE	The address of KLAXON dealers can be found on our website www.klaxon-klick.com in the "Klick Stores" area.

Thank you again for having chosen our products. We wish you a good day and enjoyable use.

Best regards

The Staff of KLAXON MOBILITY GMBH

TABLE OF CONTENTS

PRELIMINARY INFORMATION	PAGE 4
1. WARNINGS	PAGE 4
2. SAFETY NOTICES	PAGE 5
3. INTENDED USE OF THE DEVICE	PAGE 6
4. USAGE ENVIRONMENT	PAGE 6
5. OPERATING LIMITS	PAGE 6
6. USER REQUIREMENTS	PAGE 7
7. WARNINGS AND PRECAUTIONS	PAGE 7
8. MARKINGS	PAGE 11
9. PRODUCT	PAGE 12
9.1 TWIST	PAGE 12
9.2 CONTROLLER	PAGE 17
9.2.1 <i>INSTALLING THE CONTROLLER ON THE WHEELCHAIR</i>	PAGE 18
9.2.2 <i>INSTALLING THE CONTROLLER WITH VELCRO ATTACHMENT SYSTEM</i>	PAGE 19
9.2.3 <i>CONTROLLER'S USER MANUAL</i>	PAGE 20
9.3 HANDLEBAR ACCESSORY FOR RIDING WITH THE HANDLEBAR	PAGE 23
9.3.1 <i>BRAKE LEVER AND PARKING BRAKE</i>	PAGE 26
9.3.2 <i>OPENING AND CLOSING THE HANDLEBAR ACCESSORY</i>	PAGE 27
9.4 TWIST'S FRONTAL INSTALLATION KIT - CENTRAL LINKING SYSTEM KLAXON®	PAGE 29
9.4.1 <i>LINKING STEPS - CENTRAL LINKING SYSTEM KLAXON®</i>	PAGE 30
9.4.2 <i>UNLINKING STEPS - CENTRAL LINKING SYSTEM KLAXON®</i>	PAGE 32
9.5 TWIST'S FRONTAL INSTALLATION KIT - LATERAL LINKING SYSTEM KLAXON®	PAGE 33
9.5.1 <i>LINKING STEPS - LATERAL LINKING SYSTEM KLAXON®</i>	PAGE 34
9.5.2 <i>UNLINKING STEPS - LATERAL LINKING SYSTEM KLAXON®</i>	PAGE 36
9.6 TWIST'S REAR INSTALLATION KIT - REAR LINKING SYSTEM KLAXON®	PAGE 37
9.6.1 <i>TWIST'S REAR INSTALLATION KIT - VARIANT: REAR LINKING SYSTEM KLAXON® FOR FOLDABLE WHEELCHAIRS</i>	PAGE 37
9.6.2 <i>LINKING STEPS - REAR LINKING SYSTEM KLAXON® FOR FOLDABLE WHEELCHAIRS</i>	PAGE 38
9.6.3 <i>UNLINKING STEPS - REAR LINKING SYSTEM KLAXON® FOR FOLDABLE WHEELCHAIRS</i>	PAGE 40
10. HOW TO INSTALL THE TWIST TO THE FRONT OF THE WHEELCHAIR	PAGE 41
10.1 <i>ENGAGEMENT OF THE TWIST WHEN INSTALLED AT THE FRONT</i>	PAGE 41
10.2 <i>DISENGAGEMENT OF THE TWIST WHEN INSTALLED AT THE FRONT</i>	PAGE 45
11. HOW TO MOUNT THE HANDLEBAR ACCESSORY	PAGE 46
11.1 <i>INSTALLATION OF THE HANDLEBAR WITH THE TWIST ALREADY INSTALLED TO THE WHEELCHAIR</i>	PAGE 47
11.2 <i>HOW TO REMOVE THE HANDLEBAR WHEN THE TWIST IS STILL INSTALLED TO THE WHEELCHAIR</i>	PAGE 49
11.3 <i>LINKING THE TWIST AT THE FRONT WITH THE HANDLEBAR ALREADY INSTALLED</i>	PAGE 50
11.4 <i>UNLINK THE TWIST FROM THE WHEELCHAIR WHILE KEEPING THE HANDLEBAR INSTALLED</i>	PAGE 54
12. INSTALLATION OF THE TWIST AT THE REAR OF THE WHEELCHAIR	PAGE 56
12.1 <i>MANUAL LINKING OF THE TWIST TO THE REAR OF THE WHEELCHAIR</i>	PAGE 58

KLAXON Mobility GmbH - EN

12.2 AUTOMATIC LINKING OF THE TWIST TO THE REAR OF THE WHEELCHAIR	PAGE 60
12.3 UNLINKING OF THE TWIST WHEN INSTALLED TO THE REAR OF THE WHEELCHAIR	PAGE 61
13. IMPORTANT NOTES AND PRECAUTIONS CONCERNING THE USE OF THE TWIST	PAGE 62
14. DRIVING THE TWIST	PAGE 64
14.1 MANAGEMENT (ON/OFF) OF THE STEERING DIRECTIONALITY CONTROL SYSTEM	PAGE 64
14.2 DRIVING THE TWIST WITH THE CONTROLLER WHEN INSTALLED AT THE FRONT	PAGE 65
14.3 DRIVING THE TWIST WITH THE HANDLEBAR WHEN INSTALLED AT THE FRONT	PAGE 67
14.4 DRIVING WITH THE TWIST INSTALLED AT THE REAR	PAGE 68
14.5 USE OF THE TWIST DEVICE BY A THIRD PERSON	PAGE 70
15. INSTRUCTIONS FOR MAINTENANCE AND SERVICING	PAGE 71
15.1 CHARGING THE TWIST BATTERY	PAGE 71
15.2 CHARGING THE CONTROLLER OR THE HANDLEBAR CONTROL'S BATTERY	PAGE 71
15.3 TYRE INFLATION	PAGE 72
15.4 TYRE AND INNER TUBE REPLACEMENT	PAGE 72
15.5 REPLACEMENT OF COMPONENTS	PAGE 72
16. INSTALLATION	PAGE 73
17. MAINTENANCE	PAGE 73
17.1 PERIODICAL MAINTENANCE	PAGE 73
17.2 MALFUNCTIONS	PAGE 74
17.3 SERVICE LIFE OF THE PRODUCT	PAGE 75
18. BATTERY INFORMATION	PAGE 75
18.1 TRANSPORTATION, STORING AND GENERAL INFORMATION	PAGE 75
19. TECHNICAL DATA	PAGE 76
19.1 CLASSIFICATIONS OF THE DEVICE	PAGE 76
19.2 PERFORMANCE	PAGE 76
19.3 DIMENSIONAL DATA	PAGE 77
19.4 ENVIRONMENTAL OPERATING CONDITIONS	PAGE 77
19.5 TRANSPORTATION AND STORAGE ENVIRONMENTAL CONDITIONS	PAGE 77
19.6 TECHNICAL DESCRIPTION OF BATTERY CHARGER	PAGE 78
19.7 TECHNICAL DESCRIPTION OF THE BATTERY	PAGE 78
19.8 MAIN FEATURES	PAGE 78
20. CLEANING	PAGE 78
20.1 DISINFECTION	PAGE 79
21. FINGER TRAPS AND SQUEEZING POINTS	PAGE 79
22. DISPOSAL	PAGE 80
23. PRODUCT WARRANTY	PAGE 80
24. FREE WHEEL/PUSH OPTION	PAGE 80
25. PROPERTY TRANSFER	PAGE 81
26. ORIGINAL TEXT	PAGE 81

PRELIMINARY INFORMATION

INTRODUCTION

This End User's Manual explains how to use the KLAXON device called TWIST suitable for increasing the mobility of people with disabilities. The KLAXON product described in this manual is as follows:

- TWIST - Firmware: FTD_30_WHEELCHAIR_FW-V3.1.0

MANUFACTURER'S IDENTIFICATION DATA

KLAXON MOBILITY GMBH
Industriestrasse 1
9601 - Arnoldstein (Austria)
Tel.: +43 (0)664 4681294
e-mail: info@klaxon-klick.com
Hereinafter: **KLAXON**.

REVISION INFORMATION

REV.	Description of the change	DATE
01	Initial release	2022.07.07

All information, illustrations and specifications in this document are based on the latest product information available at the time of publication.

This document is subject to change due to technical innovations, without prior notice.

1. WARNINGS

- Read these instructions carefully before using the device.
- KLAXON is not liable for any failure of the end user to comply with the contents of this document.
- The use of the KLAXON TWIST device is specifically dedicated to persons with disabilities who are using a manual wheelchair.
- The use of KLAXON TWIST device in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which it is used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
- If any anomalies that may endanger the user are found during the usage: do not use the device and promptly contact KLAXON's technical support.
- Non-original spare parts and accessories have not been tested by the manufacturer. Therefore we cannot certify that these components are compliant with the needed performance and safety requirements.
- KLAXON is not responsible for any damage caused by the use of non-original spare parts or accessories.
- The installation of the product and its maintenance must be carried out exclusively by authorised KLAXON technicians, who will issue a declaration of compliance. The user must request the declaration of compliance during the first activation and after every maintenance. If the device does not comply with the declaration, the warranty will expire with immediate effect and KLAXON will not be liable for any damage resulting from the use of the device.
- Not all the provisions included in this User's Manual are valid for all the countries and for all the KLAXON products. Please always refer to the local regulations, settings and models that are provided and applied in the country of purchase and of use.

KLAXON Mobility GmbH - EN

The points of the manual marked with the following symbols should be read carefully:

	Generic DANGER sign: it indicates the existence of a danger that can derive from various causes. The word DANGER is used to describe a potential situation that, if not avoided, could produce death, serious injuries, and/or property damages.
CAUTION	The word CAUTION indicates a potentially hazardous situation, which, if not avoided, may cause minor injuries or property damage.
NOTE	The word NOTE indicates important instructions or special information.
	High voltage DANGER sign: indicates the presence of high voltages.
	Burn hazard DANGER sign: indicates the risk of burn injuries.
	Sign indicating the warnings regarding the disposal of components.

2. SAFETY NOTICE

NOTE	KLAXON devices meet all the requirements of ISO 7176-14.
	The user must read this document entirely and have become familiar with its contents before using the device.
	If the device is not used in accordance with the given specifications, the manufacturer's expected level of safety may decrease.
	During the initial period of use the user must familiarize with operation to avoid potential risks caused by incorrect use.
	EMC - The device complies with ISO 7176-21:2009 CLAUSE 5.4 standards on electromagnetic compatibility, but to ensure the safety of the user, it is recommended not to use the device in presence of strong environmental electromagnetic fields.
	EMC - The device complies with ISO 7176-21:2009 CLAUSE 5.4 standards on electromagnetic compatibility. Despite this, during its operation, the device may affect other devices with its electromagnetic field.
	Children, incapacitated persons, or those that are insufficiently familiar with the device or are physically unsuited, must not use it.
NOTE	For technical and safety information, please contact a KLAXON dealer. The addresses of the KLAXON dealers can be found on the website www.klaxon-klick.com in the "Klick Stores" area.

- The device may skid on wet pavement, on gravel or on uneven terrain: always adjust the speed and driving style to the conditions (weather, surface, individual capacity, etc.); never drive straight towards an object, and drive carefully towards narrow passages.
- On wet or uneven terrain, the value of the maximum rated slope allowed inclination and the value of the maximum allowed step height are decreased in order to safeguard the safety of the user.
- In the event of a collision, the user could suffer serious injuries to its body parts.

- At high speed the user may lose control of the wheelchair and it may tip over. Never drive faster than 20 km/h (6 Km/h in Controller mode), both uphill and downhill, and avoid impacts in general.
- Always use the TWIST-equipped wheelchair appropriately. Always brake in order to avoid obstacles (steps, edges, door frames, etc.), during descents, turns, and/or dangerous inclines.
- Driving performance of the TWIST-equipped wheelchair may be affected by electromagnetic fields. If a malfunction occurs in these conditions please contact KLAXON or one of its authorised resellers.
- Avoid contact with liquids leaking from the battery in case the battery is damaged.
- While using the device, some system components can get very hot (e.g.: braking system, etc.). A contact of these components with the skin may cause scalding and burns: be extremely careful to avoid contact.
- Always observe the operating limits for the manual wheelchair on which the TWIST device is installed.

3. INTENDED USE OF THE DEVICE

The TWIST device can be used to add an auxiliary power assist system to most of the manual wheelchairs, quickly and easily.

By installing the TWIST device at the front of the wheelchair (front configuration with or without handlebar), the front castors are lifted off. The single wheel of the "traction unit" is then used for steering.

The system consisting of the TWIST device installed to the manual wheelchair falls within Class B according to EN 12184:2014. TWIST devices meet the requirements of Class B. In accordance with the definition of Class B in EN 12184:2014, the device is intended for use in an indoor environment and is capable of dealing with outdoor obstacles.

Any other use or misuse could lead to hazardous situations.

4. USAGE ENVIRONMENT

Operating Environment

The TWIST device is manufactured in order to be installed to the majority of wheelchairs on the market. TWIST can be used both indoors and outdoors, within the limits stated in this document and, in general, according to the same limitations of use as those of the manual wheelchair (as established by its manufacturer). In case of doubt please verify all the limitations by reading the End User Manuals or by contacting the manufacturers concerned.

Medical use and types of users

The TWIST device is not intended for a specific clinical use, but as a mobility support for people using manual wheelchairs. Therefore, no professional, technical, or attitudinal requirements are necessary to use or operate the TWIST device, other than to follow the instructions described in this document. Limitations for specific categories of users are described in this End User Manual.

The TWIST device is defined under EU 745/2017-MDR as a "Class I Medical Device".

5. OPERATING LIMITS

The device must be used consistently with the intended use described in the preceding chapters, any other use is considered as improper.

In particular do not use TWIST:

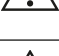
- If not installed to a manual wheelchair.
- After intake of alcohol or if the user's psycho-physical conditions are unsuitable
- On roads with unsuitable pavement.
- In poor visibility conditions (fog, etc.).
- If the user weighs over 120 kg or is heavier than allowed for the wheelchair used.
- In adverse weather (rain, strong wind, etc.).
- When going uphill or downhill on slopes that are exceeding 6°/ 10%.
- In overcoming, either in going up or down, steps higher than 30 mm in height.

6. USER REQUIREMENTS

The TWIST device and its accessories can be used autonomously by a user:

- that is informed and has had sufficient practice in the proper use of the device.
- who is aware of the risks associated with its use.
- who is in possession of sufficient physical and mental capabilities to ensure the appropriate use of the device in complete safety.

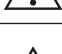
7. WARNINGS AND PRECAUTIONS

	This User's Manual is an integral part of the device and must always be available to the user. If it is lost or damaged, contact KLAXON or download the manual from the specific product webpage on www.klaxon-klick.com .
	The TWIST device and its accessories can only be used by users who are aware of the risks associated with their use.
	The TWIST device and its accessories may be used only and exclusively by a user who has been informed and has had sufficient practice in the proper use of such devices.
	Absolutely never remove, modify or replace components of the device. If necessary, these operations must be carried out only by technical personnel authorised by KLAXON.
	Never use the device after having drunk alcohol, taken drugs and in general if the user's psycho-physical conditions are unsuitable.
	The use of the TWIST device is prohibited to users weighing more than 120 kg; the total weight of the user must not exceed the amount specified by the manufacturer of the wheelchair in use.
	The user is responsible for monitoring and maintaining the efficiency of the device and its accessories in accordance to the instructions contained in this document.
	It is absolutely forbidden to use the device outside of the stated environmental operating conditions.
	Periodic maintenance must be performed to ensure proper operation and the safety of the device.
	Do not use a TWIST device to carry more than one person.
	In case the TWIST device is <u>front- or rear-installed</u> and used <u>without handlebar</u> : in such configurations, the TWIST is intended for a user who is physically and mentally capable of safely controlling the manual wheelchair in typical use situations, including inclines, using the wheelchair's handrims.
	Potential finger traps are shown in chapter 22. Potential squeezing points are shown in chapter 22.
	The TWIST device is equipped with a handle for its transport (Figure 1 Letter A). Use this handle to move and transport the device when it is not installed.

KLAXON Mobility GmbH - EN

	Do not use a TWIST device unless it is installed to a manual wheelchair.
	Do not use the supplied battery pack with devices different from the TWIST device for which it is designed.
	Do not leave the battery in the sun or adverse weather when not in use.
	Do not short-circuit the battery.
	Do not get the battery wet or immerse it in water.
	Do not attempt to open the battery pack.
	Do not allow children to play with the battery.
	Do not allow children to use the TWIST device.
	If the battery has reached the end of its life, it must be disposed of in accordance with the local regulations of the country in which it will be disposed of.
	While using the device, some system components can get very hot (e.g.: braking system, etc.). A contact of these components with the skin may cause scalding and burns: be extremely careful to avoid contact.
	The components can become very hot if exposed to the sunlight. Skin contact may cause scalding and burns: be extremely careful to avoid contact.
	DANGER: The voltages inside the device can be lethal.
	DANGER: Do not connect/disconnect power cables when the device is on.
	DANGER: Any maintenance and/or cleaning operations performed on the device must be carried out with the device switched off and disconnected from the mains.
	DANGER: Do not remove any of the unit's cover panel, contact the Technical Support Service.
	The use of the TWIST device in public spaces (streets, sidewalks, squares, cycle paths, etc.) must comply with the specific regulations of the country in which it is used. KLAXON is not liable for use that is not in accordance with specific regulations in force.
	Adjust the speed in relation to the characteristics of the wheelchair, the road conditions, the lateral inclination of the road and the radius of the curves to be negotiated.
	It is forbidden to use the Twist simultaneously with other propulsion system.

KLAXON Mobility GmbH - EN

	The TWIST device is equipped with a front light, however it is recommended to take the utmost care when driving in poor lighting conditions. Driving in poor lighting conditions is prohibited if the front light is not working or the lighting conditions are insufficient (even if the front light is on). Driving in poor lighting conditions is only possible if permitted by local country regulations and in accordance with applicable laws.
NOTE	The front light can only be operated via Web App, the front light only works when the TWIST device is installed on the front.
	The TWIST device is equipped with a rear light that allows you to be visible from behind. It is recommended to pay the utmost attention when driving in poor lighting conditions. Driving in poor lighting conditions is prohibited if the rear light is not active. Driving in poor lighting conditions is only possible if permitted by local country regulations and in accordance with applicable laws.
NOTE	The rear light can only be operated via Web App, The rear light only works when the device is installed on the rear.
	Pay attention to side-skidding due to steering while moving.
	Pay attention to side-skidding due to the slope of the road.
	Do not use the TWIST device on roads with unsuitable surfaces such as, for example, off-road tracks or in the presence of sand or mud. Do not drive through deep puddles (see chapter 13).
	Pay attention to side-skidding due to strong wind.
	Do not use the TWIST device in adverse weather conditions (rain, strong wind, etc.).
	Please bear in mind that traction and stability may be affected by the pavement and environmental conditions even within the maximum rated slope value (6°/10%).
	It is recommended to go downhill at no more than two thirds of maximum speed; avoid sudden braking or acceleration when going on inclines.
	Never attempt to overcome obstacles when going uphill or downhill.
	Do not go up or down staircases.
	Do not attempt to overcome obstacles diagonally: ensure that the rear wheels always overcome the obstacle at the same time. Never stop before having completely passed the obstacle. Do not attempt to drive over obstacles higher than 30 mm.
	When the device is in motion, avoid moving the centre of gravity or moving the body abruptly.

KLAXON Mobility GmbH - EN


	It is advisable to drive through narrow passages at minimum speed and with the utmost caution.
	When using the TWIST device, the danger of tipping over when turning increases: reduce speed before making a change of direction. Only accelerate again once the curve has been exited and only when the front wheel is straight again (aligned with the Linking System).
	When using the TWIST device, it is absolutely forbidden to make a U-turns or to start from a stationary position with the handlebar turned on rated slopes of more than 3° (or less depending on the pavement conditions) and at a speed greater than 1st gear, when riding with the Controller, and 3 km/h, when riding with the handlebar, (or less depending on the pavement conditions).
	When entering a building and/or lift, always keep in mind the turning diameter of the wheelchair equipped with the TWIST device. Avoid driving situations from which it is no longer possible to exit because it is impossible to turn the wheelchair.
	It may be difficult to manoeuvre in front of a lift or in front of a building entrance because the wheelchair's turning diameter increases once the TWIST device is installed; this may be in contrast to the building regulations or the actual dimensions of the entrances.
	Absolutely never use the TWIST device along tramways, rails and metro tracks. Pay the utmost attention, when passing a pedestrian crossing in the presence of tracks, to keep the TWIST and the wheelchair wheels at a safe distance. Failure to comply with this precaution could cause serious damages to the device and to the user. The same precautions apply to water drains, manholes and all kinds of hollows along the way.
	Please take into account that the stopping distance on inclines can be significantly greater than on flat terrain.
	Switch off the device before entering and before leaving an indoor ambient.
	Make sure the surroundings are free of hazards, precipices, drains, obstacles, collision elements, people and/or animals each time the device starts up or re-starts moving.
	Avoid collisions, crashes, dangerous routes, routes near precipices, holes and/or drains and, in general, any condition that could endanger one's own safety and the safety of people, animals and/or property.
	The TWIST device can be used only with compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria. Only authorised KLAXON technicians can decide, under their own responsibility, if the specific wheelchair can be properly used with the TWIST device (according to both wheelchair features and its general condition).
	Using the device when not properly installed exposes the driver to the risk of losing control of the vehicle.
	The TWIST device may only be operated by a user who has the necessary psychophysical capabilities to use the device in total safety. The final evaluation of the user's capabilities, according to the chosen product configuration, is under the responsibility of the dealer or, if the device is supplied under prescription, of the prescribing physician. In no way can KLAXON be held responsible for an erroneous evaluation.

8. MARKINGS

The device is uniquely identified by its serial number (SN), shown on the plate of the device, attached to the motor unit near the battery. The meaning of the symbols shown in the plate are as follows:

	Manufacturer
KLAXON TWIST (KT)	Type name
	Serial number
	KT-C0250W10
	Medical Device
	Year of manufacture
	Symbol indicating to follow the instructions for use
	Symbol indicating that the wheelchair (device) is not intended to be used as a seat in a motor vehicle
	Maximum User Weight
	CE mark
	Electrical device disposal via separated collection (special disposal, electrical components)
<u>MAX SPEED</u>	Maximum speed of the device (configuration without handlebar - configuration with handlebar)
<u>RATED SLOPE</u>	Maximum inclination that can be negotiated with the device

Picture of an example of the CE Label, applied on every TWIST device:

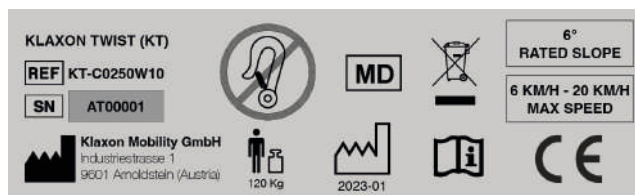9. PRODUCT

9.1 TWIST

A	KLAXON TWIST
B	Controller
C	Web App
D	Hex keys
E	TWIST Battery Charger
F	Charger for the Controller and for the Handlebar Control

NOTE	The Web App can be considered, to all practical purposes, as an accessory since the TWIST device can also be used without it.
NOTE	The Web App provides additional functionalities that are useful but not essential to operate the device. KLAXON strongly recommends the use of the Web App to enjoy a personalised driving experience.
NOTE	The Web App can be accessed and download from the following website: webapp.klaxon-twist.com . A registration is required in order to use the Web App.

Figure 1

KLAXON TWIST device components

A	Handle
B	Front light (can be activated via Web App)
C	On/off button and LED bar indicating the remaining charge level (when TWIST device is switched on)
D	Battery
E	Wheel and motor unit
F	Multi-link Klaxon®: steering unit with housing to connect driving configuration adaptations (e.g.: handlebar)
G	Upper and lower pin bracket
H	Housing for the knob screw (Figure 17 Letter K) designed to reduce the backlashes between the Linking System and the device
I	Rear light (can be activated via Web App)
J	Kickstand base

Inserting/removing the battery

Place the TWIST on a flat surface and make sure the device is switched off then:

- to insert the battery, insert it into its housing with the help of the groove on the battery casing (Figure 2)
- to remove the battery, slide it out of its housing while gripping on the groove in the battery (Figure 3)

Switching on the device and the battery

To switch on the TWIST and its battery, ensure that the latter is charged and correctly inserted in its housing, then press the power button (Figure 2 Letter A) for three seconds until the LED bar lights up. The device will emit a double acoustic signal (Beep) to confirm that it has been switched on.

Switching off the device and the battery

To switch off the TWIST device and its battery, press and hold down the switch-off button (Figure 3 Letter A) for three seconds until the LED bar goes off.

Figure 2

Figure 3

Figure 4

FIG. 3 DETAIL B: OPERATIONAL WINDOW FOR TYRE INFLATION

In order to perform maintenance operations, such as inflating the wheel, the battery must be removed from its housing on the TWIST as shown in the figure above (Figure 4).

A

B

Figure 5

FIG. 3 DETAIL C: STEERING DIRECTIONALITY CONTROL SYSTEM

The device is equipped with a spring-loaded mechanical system that allows to vary the firmness of the steering system. In position A, the system changes direction more easily. In position B, the system has greater resistance to direction changes and therefore a greater stability in rectilinear driving (Figure 5).

	<p>We recommend using the position shown in Figure 5 Letter B for uphill driving.</p>
--	---

A

B

Figure 6

DETAIL - CARRYING HANDLE

Place the KLAXON TWIST device on a flat surface, use the central groove to pull the handle out of its housing (Figure 6).

Figure 7

DETAIL - REAR LIGHT

The rear light can only and exclusively be activated via the Web App (Figure 7).

9.2 CONTROLLER

NOTE	The Controller is provided with the Quokka® attachment system. As an optional, a Velcro attachment system is available.
------	---

Figure 8

A	Stop button
B	Speed selection lever
C	Power button
D	LED bar
E	Adjustment ring for position adjustment - Version with Velcro attachment system
F	Velcro strap for securing the Controller to the wheelchair (available only on the optional Velcro attachment version)
G	Hole for Velcro insertion (available only on the optional Velcro attachment version)
H	Micro USB type B plug for charging
I	Safety loop of the Velcro strap (available only on the optional Velcro attachment version)
J	Adjustment ring for position adjustment - Version with Quokka® attachment system

Right Side Controller

If the Right Side Controller was chosen at the time of purchase, it must be installed on the right side of the wheelchair. In order to shift gears, simply push the lever forward. Conversely, to be able to downshift gears, simply push the lever backwards.

	If the Controller is installed on the opposite side to the one for which it was designated, the controls will operate in the opposite way (Figure 9). Always ensure correct installation before operating.
--	--

Figure 9

Left side Controller

If the Left side Controller was chosen at the time of purchase, it must be installed on the left side of the wheelchair. In order to shift gears, simply push the lever forward. Conversely, in order to downshift gears, simply push the lever backwards.

	If the Controller is installed on the opposite side to the one for which it was designated, the controls will operate in the opposite way (Figure 10). Always ensure correct installation before operating.
--	---

Figure 10

9.2.1 INSTALLING THE CONTROLLER ON THE WHEELCHAIR

	If it is intended to mount the Controller on the Right, make sure that a Right Controller has been purchased. If installing the Left Controller, make sure the Left Controller has been purchased.
--	--

NOTE	For the standard Quokka® system attachment please refer to this subsection and to the manual of the product provided with the attachment. For the instructions related to the Velcro attachment please refer to this subsection and to subsection 9.2.2.
------	--

- Locate the most convenient, accessible mounting position where the Controller can be installed (Figure 11).

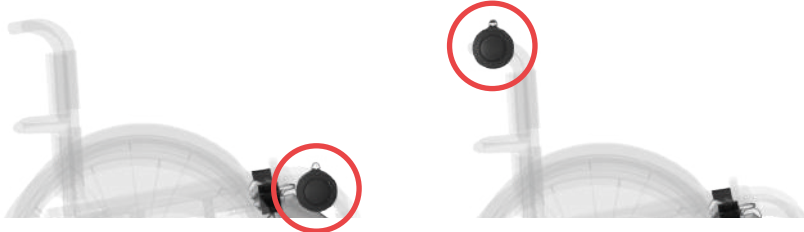
Figure 11

	It is recommended to mount the Controller in such a way that it does not interfere with or prevent the use of the wheelchair brakes.
	When mounting and/or operating any accessories or components on the wheelchair, ensure that they do not interfere with or impair the use of the Controller.
	Mount the Controller in such a way that no controls, levers and/or buttons can be operated unintentionally (including those on the wheelchair and all the accessories).

Figure 12

- Once the position has been found, screw the adjustment ring (for Velcro or Quokka®) to the Controller using the 4 screws and the supplied hex key (Figure 12).

9.2.2 INSTALLING THE CONTROLLER WITH VELCRO ATTACHMENT SYSTEM

Figure 13

- Wrap the wheelchair tube with the Velcro tape (Figure 13).
- Pull the Velcro tight and fold it over its counterpart, securing it

- Pass the head of the Velcro tape through the safety loop to prevent the Velcro from accidentally slipping off. If necessary, cut off the excess of Velcro, taking care not to shorten it too much. The folded Velcro should adhere to the counterpart wrapped around the tube, along its entire length. Excessively short Velcro may not guarantee a good mechanical hold of the Controller.


9.2.3 CONTROLLER'S USER MANUAL

General information:

The Controller is supplied by KLAXON in two versions, one that can be mounted on the right side and one that can be mounted the left side of the wheelchair. At the time of purchase, the type of Controller will be decided on by taking into consideration the most comfortable usage position for the user.

	Do not mount the Right Side Controller on the left or the Left Side Controller on the right, as the operation of the speed selection lever would work in the opposite direction.
---	--

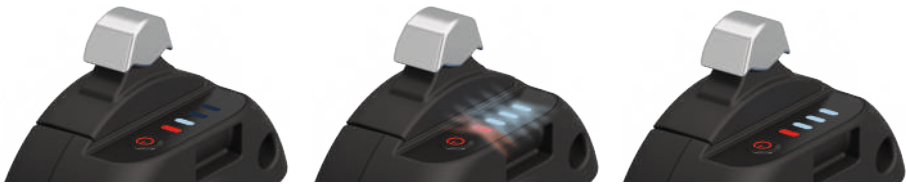The Controller is equipped with a battery that guarantees approximately 15 hours of use. The LED bar and the emission of certain acoustic signals indicate the status of the device:

A

B

C

Figure 14

- The number of lit LEDs indicates the battery charge level of the Controller (Figure 14 Letter A)
- If the LEDs are flashing, this means that the Controller is not connected to the TWIST (Figure 14 Letter B)
- Lit, non-flashing LEDs indicate that the Controller is connected and ready for use. An acoustic signal (Beep) will alert when the connection between the Controller and TWIST has been established (Figure 14 Letter C)
- A long acoustic signal (Beep) followed by a short one warns you that the Controller has been disconnected from TWIST.

	The Controller must only be activated when the TWIST device is installed to the wheelchair. Otherwise the TWIST could move abruptly if accidental or intentional driving commands are given through the controls.
	Always switch off the Controller before disconnecting the TWIST from the wheelchair.
	The Controller must never be turned on if it is not connected to the wheelchair.
	It is advisable to avoid using the device when the Controller's battery is running low as, if its battery runs out, the TWIST will perform a safety stop and can then only be moved by manual push (no function is active when the Controller is switched off).
	Do not leave the Controller with its battery empty for long periods.
	Always switch off the Controller when not in use or when using other driving controls.

	Do not clean the Controller with solvents or products not recommended for plastic materials. Before applying the cleaning product, check the instructions of the chosen product to ensure that it is compatible.
	Do not wet or immerse the Controller in water.
	Use only the supplied battery charger to charge the Controller.

Controller Usage Information - Braking typologies

	The motor of the TWIST device does not perform any braking and/or blocking action on the device when it is not in motion (recorded speed approx. 0 Km/h). This also refers to the <u>Braking</u> and the <u>Temporary Braking</u> functions triggered by the Controller when the device stops (reaches approx. 0 Km/h). It is therefore necessary to act on the manual wheelchair by actively blocking its movement. It is also necessary to act on the manual wheelchair to keep it stationary in position.
---	--

Braking: A quick push to the stop button activates the Braking function of the TWIST motor that will decrease the speed until the device stops (recorded speed approx. 0 km/h) (Figure 15).

Quick push

Figure 15

	When the TWIST is installed at the front, braking will be very effective in all conditions of use even when riding downhill. When the TWIST is rear-installed the wheel will not be able to brake the device, therefore activating the <u>Braking</u> function will only stop the wheel's power output but will not effectively slow down the device (Figure 15).
	In any case, it will be necessary to act by actively braking the manual wheelchair to keep it stationary in position.

Temporary Braking: A prolonged pressure (more than 1.5 seconds) on the stop button activates the Temporary Braking function of the motor; the device will slow down to a stop (approx. 0 km/h) as in the Braking function but, when the button is released, the motor will restart moving at the minimum speed (1st gear) of approx. 2 km/h (Figure 16).

Keep the pressure on the button for more than 1.5 seconds

Figure 16

	<p>When the TWIST is installed at the front, braking will be very effective in all conditions of use even when riding downhill. In case the TWIST is installed at the rear, the wheel will not be able to brake the TWIST; therefore, activating the <u>Temporary Braking</u> function will only ensure that the wheel's power output is stopped, but will not brake the wheelchair (Figure 16). If it is necessary to remain stationary in position before the restart, act by actively braking the manual wheelchair.</p>
	<p>If the <u>Temporary Braking</u> function is activated by mistake, press the stop button immediately and quickly to execute the <u>Braking</u> function. It will then be necessary to act by actively braking the manual wheelchair to keep it stationary in place.</p>

Controller usage information - Speed management

A first quick forward push and the subsequent release of the speed selection lever sets a travel speed of approximately 2 km/h (1st gear).

	<p>Caution: the speed will only be set when the speed selection lever is released.</p>
---	--

A second quick forward push and the subsequent release of the speed selection lever sets a travel speed of approximately 3.5 km/h (2nd gear).

	<p>Caution: the speed will only be set when the speed selection lever is released.</p>
---	--

A third quick forward push and the subsequent release of the speed selection lever sets a travel speed of approximately 4.5 km/h (3rd gear).

	<p>Caution: the speed will only be set when the speed selection lever is released.</p>
---	--

A fourth quick forward push and the subsequent release of the speed selection lever sets a travel speed of approximately 6 km/h (4th gear).

	<p>Caution: the speed will only be set when the speed selection lever is released.</p>
---	--

It is possible to shift up and down quickly from 1st to 4th gear by quickly giving 4 quick push in sequence.

To reduce the speed, apply a quick push on the speed selection lever in the opposite direction.

	<p>Caution: the speed will only be set when the speed selection lever is released.</p>
---	--

Each action in the Controller will be matched by an acoustic signal from the TWIST confirming that the command has been received.

The acoustic warning can be disabled using the dedicated Web App on a Smartphone.

	<p>When the TWIST device and the wheelchair are being controlled by a third person, that person must have read and understood this document in its entirety and must comply entirely with the user's requirements.</p>
	<p>When the TWIST device and the wheelchair are controlled by a third person, this person must always ensure that it has the psycho-physical ability to control the TWIST device, to operate the manual wheelchair and to brake/stop both the TWIST device and the wheelchair. If in doubt, please consult all the available manuals for all the aids used, including the wheelchair, and/or to undertake the appropriate training to use them.</p>

	When the TWIST device and the wheelchair are controlled by a third person, this person must guarantee that it can always control, and if necessary brake/stop, the TWIST device and the manual wheelchair at all times. This person is also responsible for choosing the route, driving correctly, controlling the suitability of the route, the pace and of the braking/stopping of the device and of the wheelchair, as well as for everything that happens to the people, objects, animals and the environment involved as long as the TWIST device is installed to the wheelchair.
	It is forbidden, under any circumstances, that the control of the device should be transferred/ assigned to persons who do not meet the requirements stated in this document.
	It is forbidden, under any circumstances, to leave the controls in such a way that its operator cannot access them at any time during use.

In case of Controller connection problems

In the event that the Twist does not restart moving, stops or does not execute commands make sure to be in a location away from danger and follow the following steps: Press the Stop button. If the problem persists: switch the Controller off and on again. If the problem persists: switch the Controller and Twist off and on again. If the above steps do not solve the problem: do not use the device and contact an authorised KLAXON service centre.

9.3 HANDLEBAR ACCESSORY FOR RIDING WITH THE HANDLEBAR

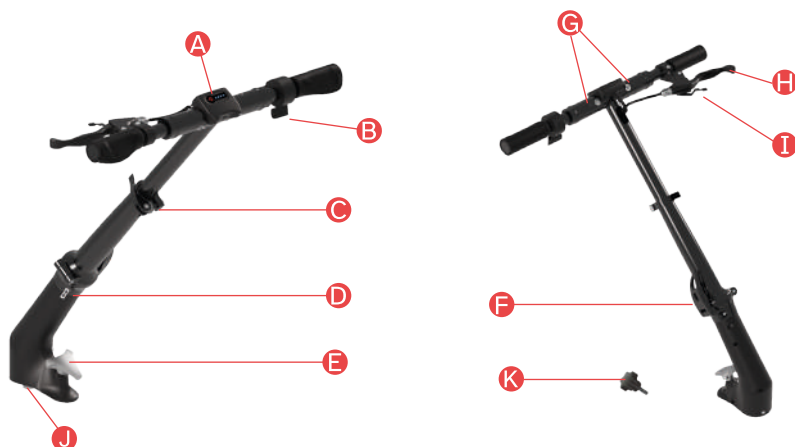
Figure 17

A	Handlebar Control with power button and LED bar
B	Accelerator
C	Release lever for handlebar height adjustment
D	Alignment arrows for matching the upper and lower handlebar components
E	Knob for fixing the handlebar to the TWIST device
F	Release lever to fold the handlebar
G	Right and left cuff to be operated in order to fold the upper part of the handlebar
H	Mechanical and electronic brake lever
I	Parking brake
J	Steering system hub
K	Knob screw designed to reduce the backlashes between the Linking System and the device

Figure 18

Positioning the handlebar on the TWIST (Figure 18). See chapter 11 for the complete procedure.

Figure 19

Handlebar installed on the TWIST (Figure 19).

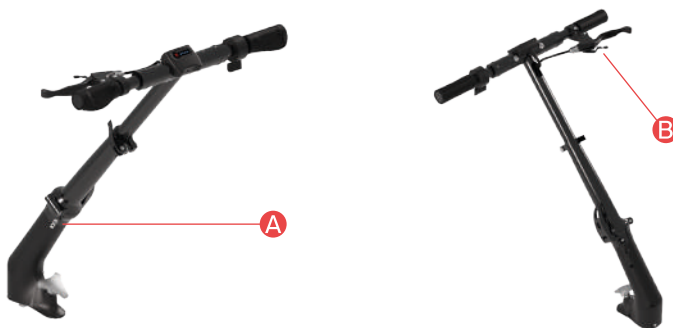
Figure 20

Figure 21

DETAIL 1 (FIGURE 20 A)

Detail of the alignment arrows between the lower handlebar component and the upper handlebar component..


Figure 22

DETAIL 2 (FIGURE 20 B)

Detail of the parking brake on the brake lever.

9.3.1 BRAKE LEVER AND PARKING BRAKE

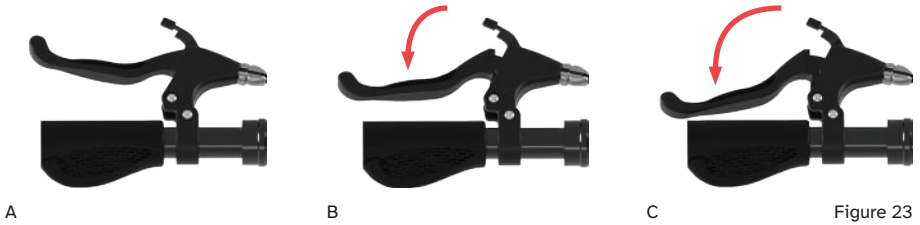Connecting the handlebar to the TWIST device enables the mechanical brake and parking brake of the device..


Mechanical Brake and Electronic Brake System (EBS)

- Brake lever (Figure 23 Letter A).
- A light pressure on the brake lever will only and exclusively activate the electronic EBS brake (Figure 23 Letter B).
- Applying more force to the brake lever will also activate the mechanical brake (Figure 23 Letter C).

	<p>The EBS brake will slow the device until it stops (recorded speed approx. 0 km/h) but will not hold the device and wheelchair in place. Act on the mechanical brake on the handlebar or on the wheelchair to keep the TWIST and wheelchair in position.</p>
--	--


Parking Brake

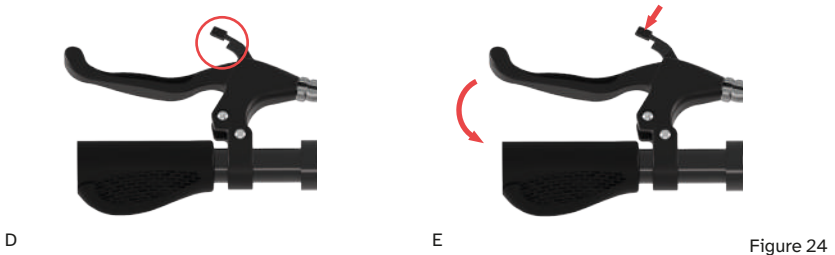Here below the instructions for enabling the parking brake on the handlebar accessory:

- Press the brake lever.
- Lower the parking brake lever (Figure 24 Letter E).
- Release the brake lever after ensuring that the parking brake is correctly engaged (Figure 25 Letter F).

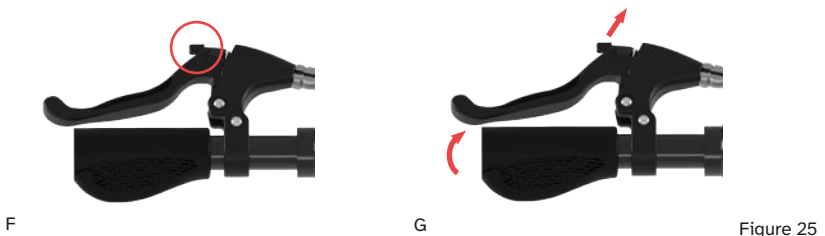Below the instructions for disabling the parking brake on the handlebar accessory:

- Press the brake lever until the parking brake lever is free to move.
- Raise the parking brake lever without letting go of the brake lever (Figure 25 Letter G).
- Release the brake lever which will return to its natural position as shown in Figure 24 Letter D.

9.3.2 OPENING AND CLOSING THE HANDLEBAR ACCESSORY

The handlebar of the TWIST is designed so that it can be folded or unfolded in 4 moves.

Figure 26

A	Right cuff for folding the upper part of the handlebar
B	Left cuff for folding the upper part of the handlebar
C	Lever for adjusting the length of the upper part of the handlebar
D	Central joint
E	Brake cable

- Pull the cuff (Figure 26 Letter A) on the accelerator side of the handlebar (Figure 27 Letter A).
- Pull the cuff (Figure 26 Letter B) on the brake-side of the handlebar (Figure 27 Letter B).

A

B

Figure 27

- Bend the two upper handlebars by 90° (Figure 28).

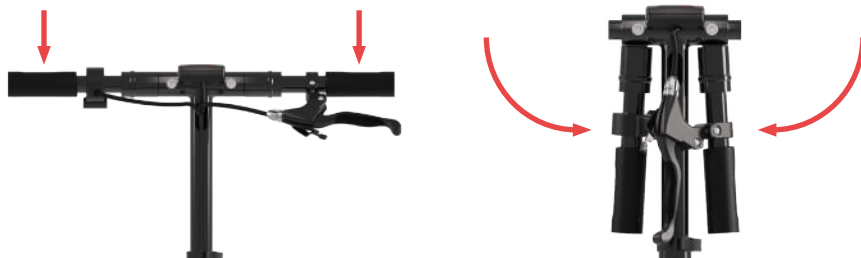
Figure 28

- Release the fixing lever (Figure 26 Letter C) and lower the handlebar into its seat to shorten it (Figure 29).

Figure 29

- Re-engage the lever (Figure 26 Letter C)
- Disarm the central joint lever (Figure 26 Letter D)
- Slowly fold the two parts of the handlebar, while taking care to accompany the brake cable sheath (Figure 26 Letter E), which is located inside it, avoiding straining it during folding.
- To reopen the entire handlebar, perform the same operations in reverse. Always take great care not to bend the brake cable sheath, and to accompany it everytime during the operation.

	Once the handlebar has been mounted, always check that the mechanical brake is working properly before using the device.
	If the brake sheath is strained during the opening and closing operations of the handlebar, there is a risk of damaging the mechanical brake, and rendering it inoperative. If this happens, do not use the device and contact a KLAXON service centre.
	If the brake cable is damaged or does not work, the mechanical brake may not work. Do not use the device and contact a KLAXON service centre.
	The adjustment of the brake lever may only be carried out by an authorised KLAXON technician. An overtightened brake can cause the vehicle to roll over if the brake lever is squeezed too hard.

9.4 TWIST'S FRONTAL INSTALLATION KIT - CENTRAL LINKING SYSTEM KLAXON®

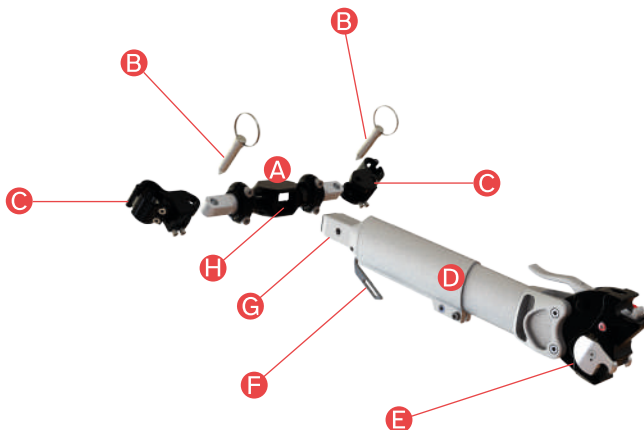
Figure 30

A	Crossbeam
B	Locking Pins
C	Clamping rings
D	Connector
E	Front Hook 2.0 Klaxon®
F	Connector's lock/unlock lever
G	Mounting system in the crossbeam (with ball type lock)
H	Square seat for the insertion of the crossbeam

	<p>The Clamping rings (Figure 30 Letter C) are a component that remains mounted on the wheelchair frame and cannot be installed/removed except by an authorised KLAXON dealer.</p>
	<p>The Linking Systems can only be installed on compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria. The Linking Systems must be installed only by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with that Linking System (according to both wheelchair features and its general condition).</p>

9.4.1 LINKING STEPS - CENTRAL LINKING SYSTEM KLAXON®

	<p>Throughout the linking phase, make sure to be on a flat area, as free as possible of unevenness in the terrain and away from dangers to the safety of the person and of others.</p>
	<p>Be careful not to squeeze or entrap your fingers during the linking operations. Potential finger traps and squeezing points are shown in chapter 21.</p>

- Insert the crossbeam (Figure 30 Letter D) in one of the two clamping ring (Figure 30 Letter C) and secure it by inserting the specific locking pin (Figure 30 Letter B). (Figure 31).

Figure 31

- Insert the crossbeam also in the second clamping ring and secure it by inserting the specific locking pin (Figure 32).

Note: The crossbeam can always remain installed on the wheelchair; if necessary (for loading and unloading from vehicles and closing the foldable wheelchair) it can be removed and placed in a safe place.

Figure 32

- Insert the connector (Figure 30 Letter D) in the square seat (Figure 30 Letter H) of the crossbeam (Figure 30 Letter A), pressing the lever (Figure 30 Letter F) located on its bottom. Once inserted, release the lever and make sure that the connector is securely locked (Figure 33).


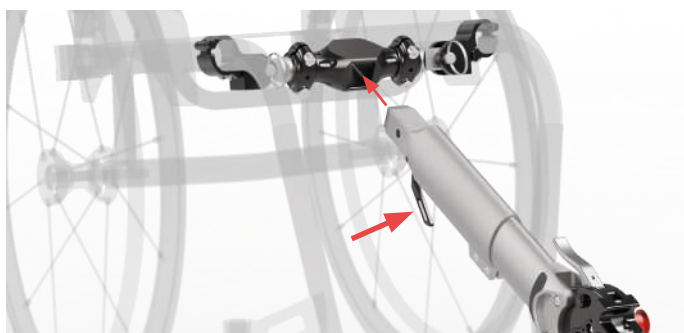
Figure 33

- Once the crossbeam is inserted, the Linking System will appear as in Figures 34 and 35.
- In order to proceed with the TWIST connection please follow the instruction from chapter 10.

Side view: Central Linking System Klaxon®

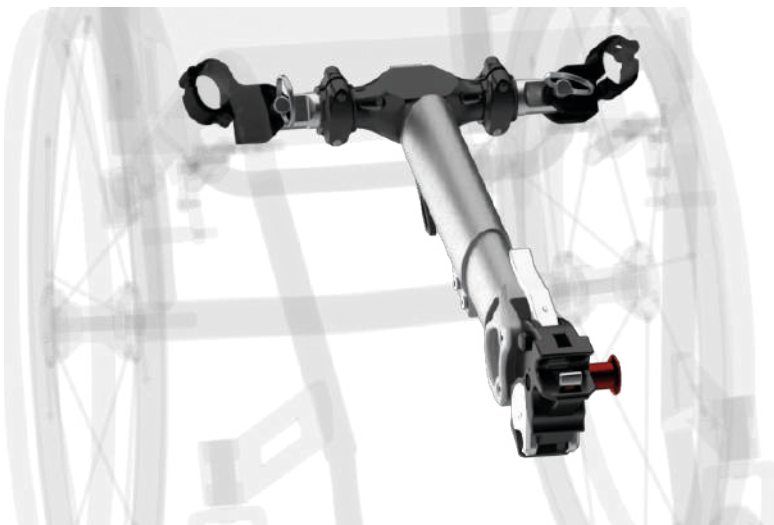Figure 34

View in perspective: Central Linking System Klaxon®

Figure 35

9.4.2 UNLINKING STEPS - CENTRAL LINKING SYSTEM KLAXON®

	Throughout the unlinking phase, make sure to be on a flat area, as free as possible of unevenness in the terrain and away from dangers to the safety of the person and of others.
	Be careful not to squeeze or entrap your fingers during the unlinking operations. Potential finger traps and squeezing points are shown in chapter 21.

- Remove the device as described in sections 10.2 and 11.4.
- Remove the connector (Figure 30 Letter D) by pressing the lever at the bottom of the connector (Figure 30 Letter F).

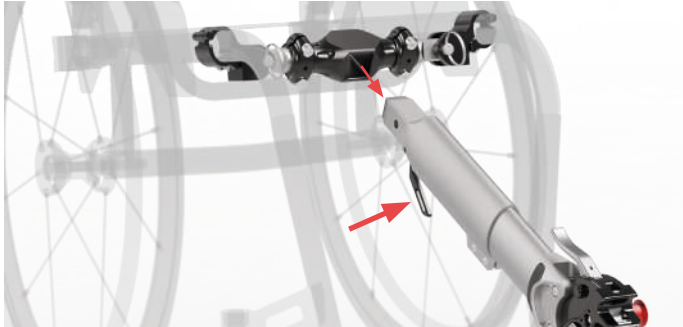
Figure 36

- Store the connector in a safe place.

9.5 TWIST'S FRONTAL INSTALLATION KIT - LATERAL LINKING SYSTEM KLAXON®

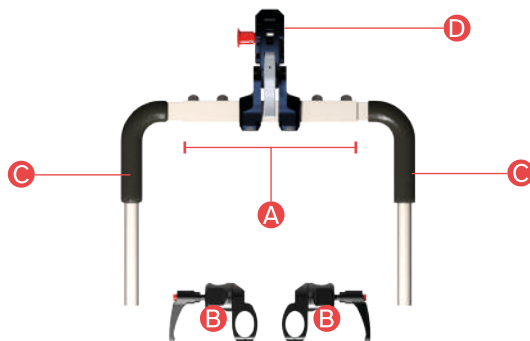
Figure 37

Lateral Linking System Klaxon® kit

A	Crossbeam
B	Clamping rings
C	Forks
D	Front Hook 2.0 Klaxon®

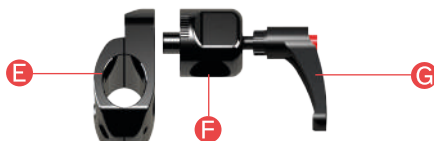
Figure 38

Elements of the Clamping Rings (details of Figure 37 Letter B)

E	Bracket
F	Clamp
G	Clamp's lever screw

	<p>The Clamping Rings (Figure 37 Letter B) are a component that remains mounted on the wheelchair frame and cannot be installed/removed except by an authorised KLAXON dealer.</p>
	<p>The Linking Systems can only be installed on compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria. The Linking Systems must be installed only by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with that Linking System (according to both wheelchair features and its general condition).</p>

9.5.1 LINKING STEPS - LATERAL LINKING SYSTEM KLAXON®

	<p>Throughout the linking phase, make sure to be on a flat area, as free as possible of unevenness in the terrain and away from dangers to the safety of the person and of others.</p>
	<p>Be careful not to squeeze or entrap your fingers during the linking operations. Potential finger traps and squeezing points are shown in chapter 21.</p>

- Loose both the clamps' lever screws (Figure 38 Letter G) enough to let the forks' tubes slide in easily and without interferences (Figure 39).

Figure 39

- Insert the forks' tubes into the clamps (Figure 38 Letter F) according to the distance indicated by the installation technician. It is important to ensure that the forks protrude from the rear of the clamps for at least 30 mm and to the same extent on both sides (Figure 40).
- Verify that the forks are correctly and evenly inserted. In case something is wrong correct the insertion and then proceed as follows (Figure 40).

Figure 40

- Tight both the clamps' lever screws until the forks are safely and strongly fixed (Figure 41).

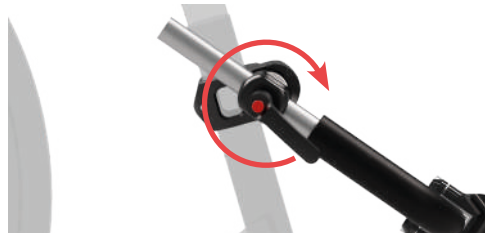
Figure 41

- Verify that the forks are correctly secured so that they cannot move and so that they cannot slide out of the clamps during use. In case the forks are not fixed enough, tighten more the clamps' lever screws until they meet the requirement then proceed as follows.
- After having inserted the forks, the device will be as shown in Figures 42 e 43.
- In order to proceed with the TWIST connection please follow the instruction from chapter 10.

Side view: Lateral Linking System Klaxon®

Figure 42

View in perspective: Lateral Linking System Klaxon®

Figure 43

9.5.2 UNLINKING STEPS - LATERAL LINKING SYSTEM KLAXON®

	Throughout the unlinking phase, make sure to be on a flat area, as free as possible of unevenness in the terrain and away from dangers to the safety of the person and of others.
	Be careful not to squeeze or entrap your fingers during the unlinking operations. Potential finger traps and squeezing points are shown in chapter 21.

- Remove the device as described in sections 10.2 and 11.4.
- Loosen the clamps' lever screws (Figure 38 Letter G) until the forks (Figure 37 Letter C) can slide out easily and without interferences (Figure 44).

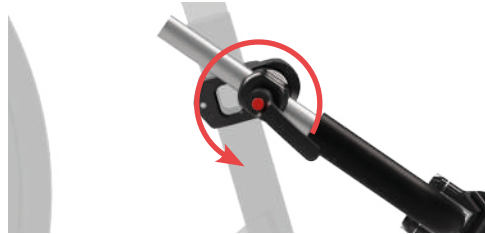
Figure 44

- Remove the Lateral Linking System kit by sliding the forks out of the clamps (Figure 38 Letter F). (Figure 45).

Figure 45

- At this point the Lateral Linking System is disconnected both from the TWIST and the wheelchair as shown in Figure 46. Please note that the clamping rings always remain installed on the wheelchair.

Figure 46

- Store the Lateral Linking System kit in a safe place.

9.6 TWIST'S REAR INSTALLATION KIT - REAR LINKING SYSTEM KLAXON®

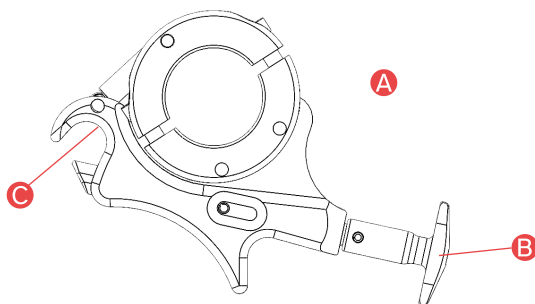
Figure 47

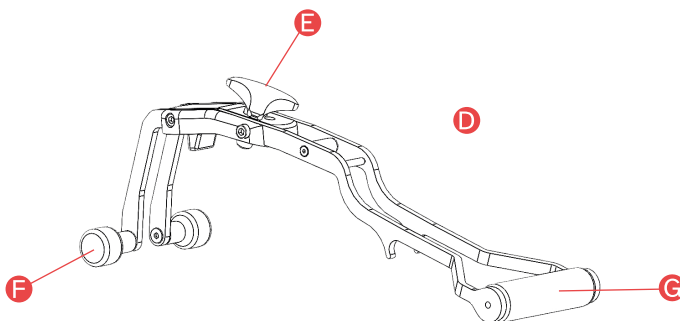
Figure 48

A	Rear linking
B	Engagement/disengagement lever
C	Seat of the upper pin
D	Frame for rear linking
E	Fixing knob
F	Front kickstand
G	Handle for transportation

The Linking Systems can only be installed on compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria. The Linking Systems must be installed only by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with that Linking System (according to both wheelchair features and its general condition).

9.6.1 TWIST'S REAR INSTALLATION KIT - VARIANT: REAR LINKING SYSTEM KLAXON® FOR FOLDABLE WHEELCHAIRS

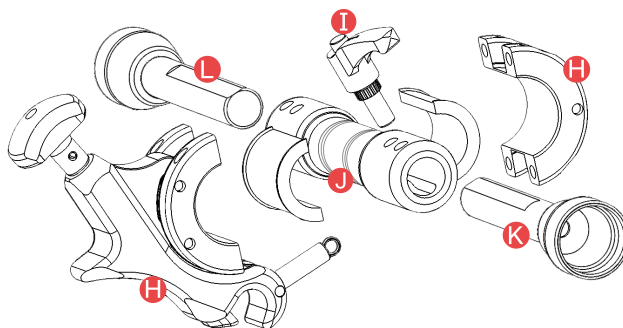
Figure 49

Rear Linking System Kit for foldable wheelchairs

H	Rear Linking (described in details in Figure 47)
I	Clamping Lever
J	Crossbeam
K	Telescopic Axle (recognisable by the presence of the Clamping Lever)
L	Fixed Axle

The Linking Systems can only be installed on compatible wheelchairs. KLAXON provides to the distributors the Installation Manual which contains the coupling criteria and the possible configurations. The Rear Linking System Kit for foldable Wheelchair can only be configured by authorised KLAXON technicians who may decide, under their own responsibility, if the specific wheelchair can be properly used with the aforementioned system (according to both wheelchair features and its general condition).

9.6.2 LINKING STEPS - REAR LINKING SYSTEM KLAXON® FOR FOLDABLE WHEELCHAIRS

Throughout the linking phase, make sure to be on a flat area, as free as possible of unevenness in the terrain and away from dangers to the safety of the person and of others.

1. Unscrew the Clamping Lever (Figure 49 Letter I) by half a turn to release the telescopic axle (Figure 49 Letter K). (Figure 50).

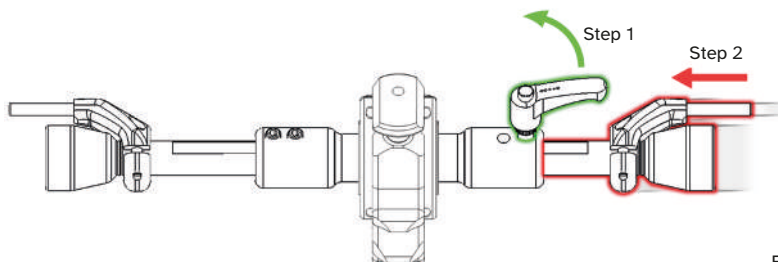
Figure 50

- Retract the telescopic axle (Figure 49 Letter K) until it reaches its minimum extension. Do not tighten the Clamping Lever before reaching step number 5 (Figure 50)
- Position the Rear Linking System Kit for foldable wheelchairs, at its minimum extension, between the wheelchair's wheel axles (Figure 51).

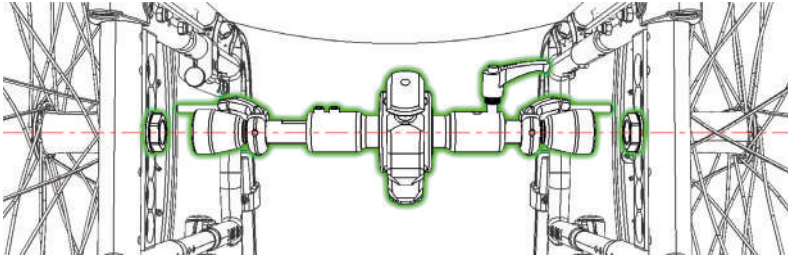
Figure 51

- Extend the telescopic axle until it reaches the maximum possible extension between the wheels of the wheelchair (Figure 52).
- Tighten the fixing lever to secure the telescopic axle in position (Figure 52).

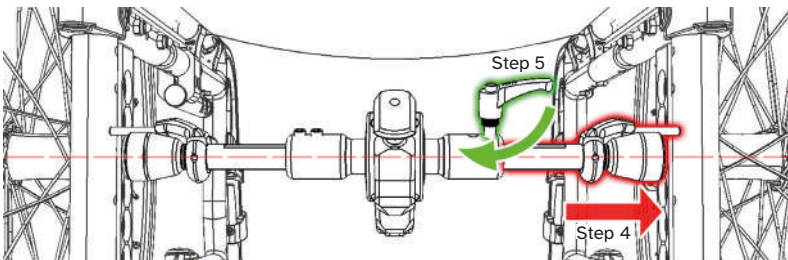
Figure 52

- Check that the Kit is securely fixed to the pins/nuts of the wheelchair. The correct fixing position is illustrated in Figure 53. Figures 54 and 55 show examples of incorrect positioning.

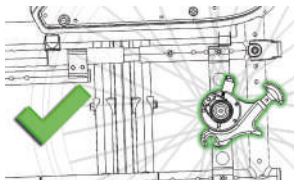
Figure 53

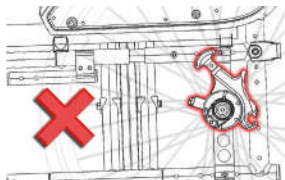
Figure 54

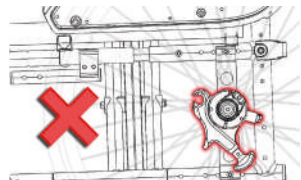
Figure 55

- In order to proceed with the TWIST connection please follow the instruction at chapter 12.

Using the device with the Kit incorrectly mounted to the wheelchair may expose to dangerous driving situations and to DANGEROUS conditions.

9.6.3 UNLINKING STEPS - REAR LINKING SYSTEM KLAXON® FOR FOLDABLE WHEELCHAIRS

Throughout the unlinking phase, make sure to be on a flat area, as free as possible of unevenness in the terrain and away from dangers to the safety of the person and of others.

1. Unscrew the Clamping Lever (Figure 49 Letter I) by half a turn to release the telescopic axle (Figure 49 Letter K). (Figure 56).
2. Retract the telescopic axle until it reaches its minimum extension (Figure 56).

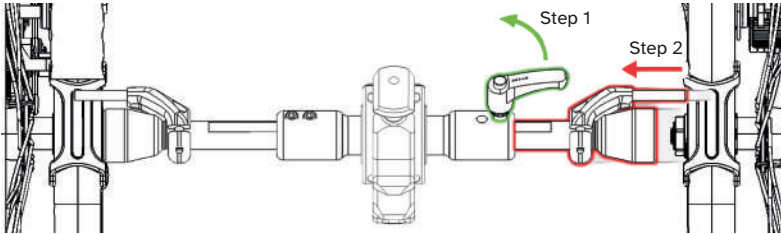
Figure 56

3. Remove the Kit from the wheelchair (Figure 57).

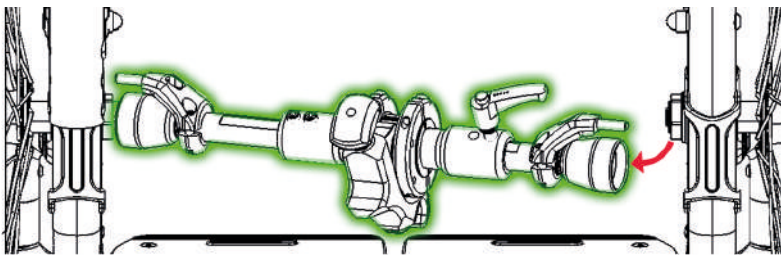
Figure 57

4. Store the Kit in a safe place.

10. HOW TO INSTALL THE TWIST TO THE FRONT OF THE WHEELCHAIR

The TWIST is designed to be used and therefore to be installed to the wheelchair in various positions on the wheelchair's frame. Each of these different applications are designed to meet precise user's needs and driving requirements.

	Always carry out the linking operation on a flat surface and if possible with the wheels of the wheelchair and of the TWIST lying on a smooth surface.
	It is very dangerous to carry out the linking operation of the TWIST to the wheelchair if the Controller/Handlebar Control is switched on. Make sure all the controls are switched off before proceeding.
	Before switching on the device make sure that the controls are mounted on the wheelchair (Controller) or on the device (Handlebar Control). Only proceed with switching on the controls when the TWIST linking procedure has been completed.
	Once the TWIST is installed to the wheelchair, make sure that only the control you intend to use is activated. Make sure that any other controls are switched off.
	It is advisable to avoid using the device when the Controller's/Handlebar Control's battery is running low as, if its battery runs out, the TWIST will perform a safety stop and can then only be moved by manual push (no function is active when the Controller is switched off).
	Before linking the TWIST, ensure that the battery is present and that it is inserted correctly and completely into its slot.
CAUTION	If the battery is not inserted in the TWIST, the latter will become unbalanced and therefore unstable on its stand.
	If the battery is not fully inserted in its housing, it may impede the rotation of the steering system.

10.1 ENGAGEMENT OF THE TWIST WHEN INSTALLED AT THE FRONT

The following instructions apply both to the front installation with the Central Linking System and to the front installation with the Lateral Linking System.

- Insert the Linking System by following the instructions in subsections 9.4.1 and 9.5.1 (Figure 58).

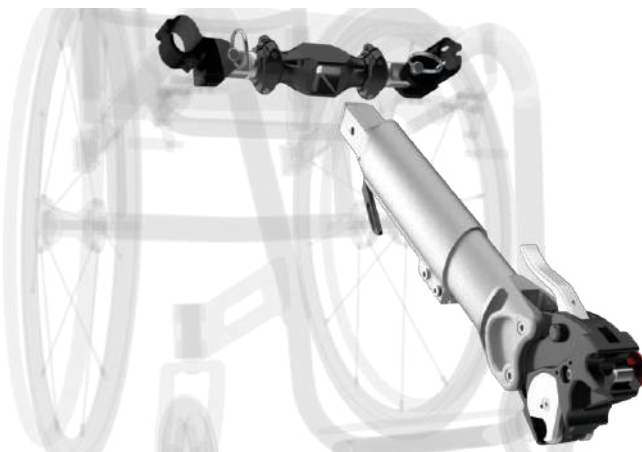
Figure 58

- Place the TWIST on the ground in front of the wheelchair and in front of the Front Hook 2.0 Klaxon®, place it on its four-legged stand (Figure 59).

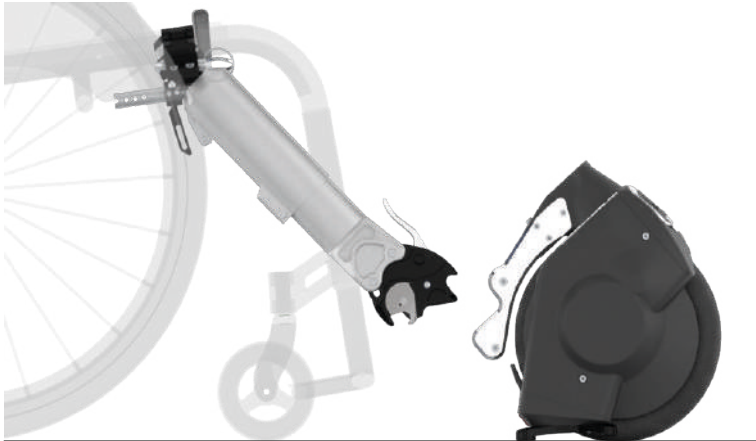
Figure 59

- Press the red button (Figure 60 Letter A) on the side of the Front Hook 2.0 Klaxon® to make sure that the hook opening is free and ready to accommodate the upper pin. If the button is locked, the system is ready to be installed (Figure 60).

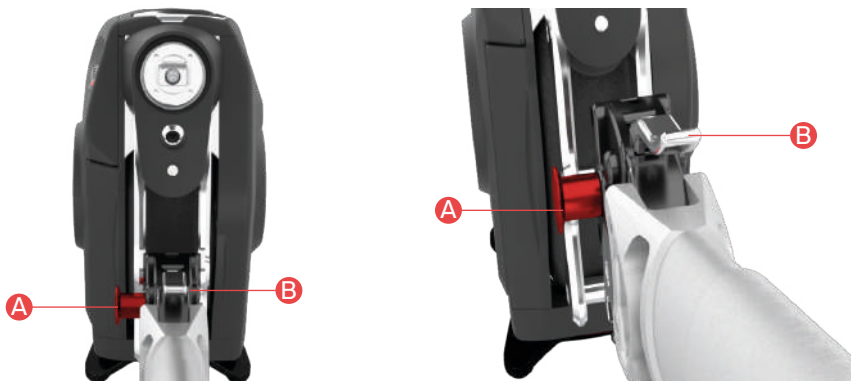
Figure 60

- Move the wheels of the wheelchair closer to the TWIST until the upper pin hook completely surrounds the upper pin (Figure 61).

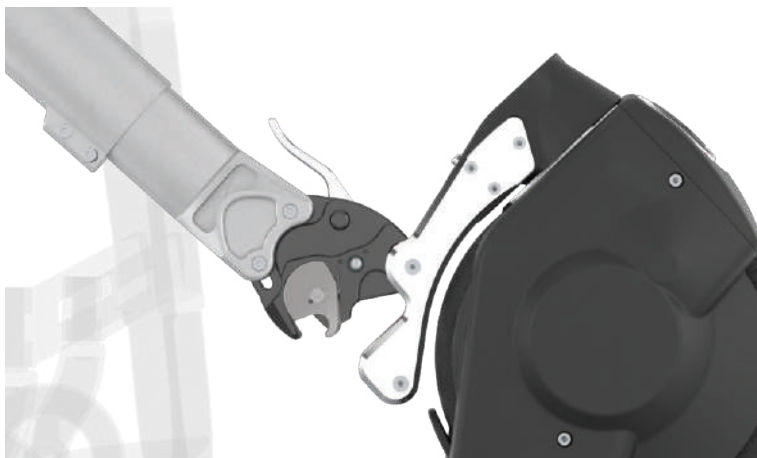
Figure 61

- If necessary and possible, use one hand to hold the TWIST in place, while ensuring that it does not move forward during the linking phase (Figure 62).

Figure 62

- Firmly lift the lever (Figure 63 Letter B) until it reaches the end of its movement and until the Front Hook 2.0 Klaxon® engages, securing the TWIST. Check that linking has been carried out correctly by checking that a green mark has appeared around the red button (Figure 63 Letter A), this will confirm that linking has been successfully completed (Figure 63). If the green mark does not appear, repeat the operation from the start.

If the green mark does not appear when the operation is repeated, this means that the Front Hook 2.0 Klaxon® is not properly engaged. Please contact a KLAXON service centre to carry out a technical check.

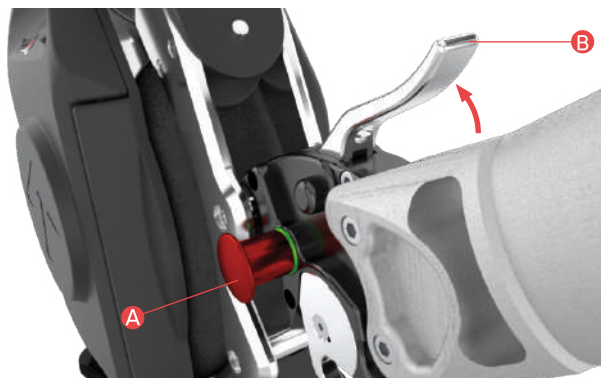
Figure 63

- Make sure that the wheels of the wheelchair are not blocked by the brakes. Push the two wheels of the wheelchair forward at the same time to establish the complete linking of the TWIST to the wheelchair and to lift up the front castors. (Figure 64)

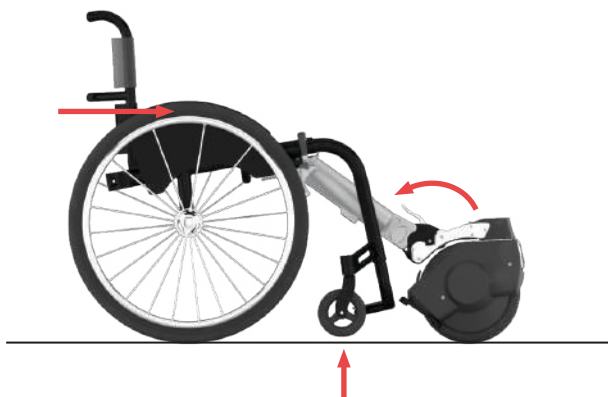
Figure 64

Once the device has been installed to the wheelchair, it is important to check that the front castors of the wheelchair are between 25 mm and 40 mm above the ground. If this requirement is not fulfilled, do not use the device under any circumstances and contact an authorised KLAXON service centre immediately.

10.2 DISENGAGEMENT OF THE TWIST WHEN INSTALLED AT THE FRONT

	<p>Always carry out the unliking operation on a flat surface and if possible with the wheels of the wheelchair and of the TWIST lying on a smooth surface.</p>
	<p>It is very dangerous to carry out the unliking operation of the TWIST to the wheelchair if the Controller/Handlebar Control is switched on. Make sure all the controls are switched off before proceeding.</p>
	<p>Make sure there are no other controls switched on when switching off the Controller/Handlebar Control.</p>
	<p>When unliking the TWIST, it is recommended to block the wheelchair's wheels using the wheelchair's brakes.</p>

- Switch off the connected wireless control.
- Lift the lever (Figure 63 Letter B) to disengage the lower pin and allow the TWIST to rest on the ground on its kickstand.

	<p>To prevent the TWIST from abruptly touching the ground during this unliking phase, the user (who must have sufficient motor skills to perform the action described as follows) can hold the TWIST with one hand and simultaneously lift the lever (Figure 63 Letter B) during the unliking phase (Figure 65).</p>
---	--

Figure 65

- Press the red button (Figure 63 Letter A) to release and unlik the TWIST from the wheelchair (Figure 66)

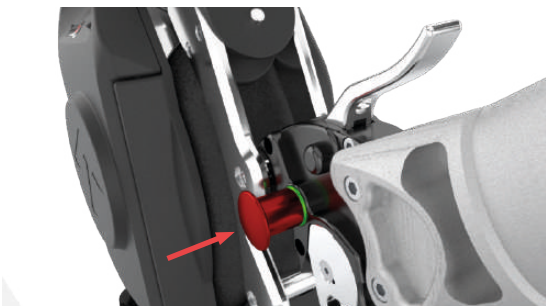
Figure 66

- Switch off the TWIST by pressing the off button on the battery. The battery, and consequently the TWIST, will switch off. In the event of a 10-minute period of inactivity, the battery and TWIST will switch off automatically.

11. HOW TO MOUNT THE HANDLEBAR ACCESSORY

Mounting the handlebar on the TWIST can be done in two ways:

- with the TWIST already installed to the wheelchair
- by mounting the handlebar on the TWIST before linking it to the wheelchair.

	It is very dangerous to mount the handlebar on the TWIST if the control is on. Make sure the Handlebar Control is deactivated before proceeding.
	Always perform this procedure with the brakes of the wheelchair engaged.
	Always carry out the mounting operation on a flat surface and if possible with the wheels of the wheelchair and the TWIST lying on a smooth surface.

General Information

The Handlebar Control (Figure 17 Letter A) is equipped with a battery that guarantees approximately 15 hours of use. The LED bar indicates the status of the Handlebar Control:

A

B

C

Figure 67

- The lit LEDs indicate the battery charge level of the Handlebar Control (Figure 67 Letter A).
- The flashing of the LEDs indicates that the Handlebar Control is not connected to the TWIST (Figure 67 Letter B).
- Lit, non-flashing LEDs indicate that the Handlebar Control is lit, connected and ready for use (Figure 67 Letter C).

	Do not leave the Handlebar Control with its battery empty for long periods.
	Always switch off the Handlebar Control when not in use, during the mounting phases on the TWIST or when using other driving controls.
	Do not clean the control with solvents or products not recommended for plastics. Before applying, check the instructions of the chosen product to ensure that it is compatible.
	Do not get the Handlebar Control wet or immerse it in water.
	Use only the supplied battery charger to charge the Handlebar Control.
	Always switch off the control before removing the handlebar and/or TWIST from the wheelchair.
	The control must never be switched on if the handlebar and TWIST are not installed to the wheelchair.

	<p>It is advisable to avoid using the device when the Handlebar Control's battery is running low as, if its battery runs out, the TWIST will perform a safety stop and can then only be moved by manual push (no function is active when the Controller is switched off).</p>
	<p>When mounting the handlebar and connecting it, make sure that no other controls in the vicinity are switched on.</p>

11.1 INSTALLATION OF THE HANDLEBAR WITH THE TWIST ALREADY INSTALLED TO THE WHEELCHAIR

- Make sure that the TWIST is correctly installed to the wheelchair and is in the correct operating position (the wheelchair should have the front castors raised off the ground). It will not be possible to mount the handlebar if the TWIST is not in the specified position (Figure 68).

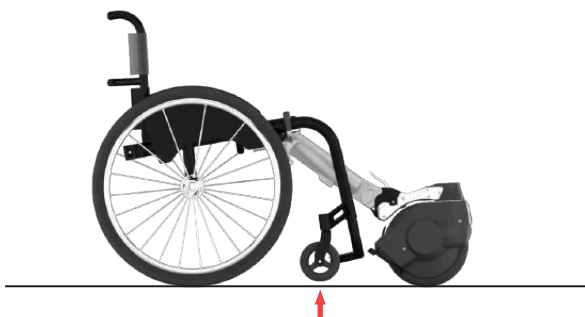
Figure 68

- Ensure that the TWIST has the wheel aligned with the handlebar. The wheel must not be steered to the right or left (Figure 69).

Figure 69

- Make sure the Handlebar Control is switched off.
- Make sure that the Controller is also switched off.
- Hold the handlebar, making sure that the two alignment arrows are aligned with each other. Only when this position is reached will it be possible to mount the handlebar (Figure 70).

Figure 70

	<p>Do not use the device with the handlebar if the arrows in Figure 70 are not aligned. In case the arrows are not aligned, the controls are not in the expected position for driving straight.</p>
---	---

- Insert the handlebar base into the steering base with a movement perpendicular to the steering system of the TWIST (Figure 71).

Figure 71

- Make sure that the handlebar is fully inserted in its housing on the steering unit (Multi-link Klaxon®) and thus in full contact with the latter (Figure 72).

Figure 72

- Screw the knob clockwise until it is as tight as possible. Caution: do not use tools, but only the hands (Figure 73).

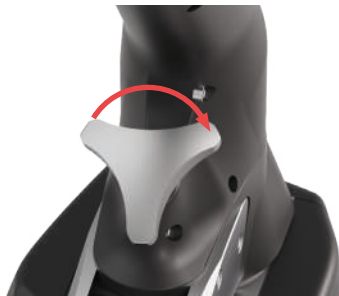
Figure 73

Using the device with the knob in Figure 73 not securely tightened exposes to the risk that the handlebar disengages with a consequent loss of control of the device.

- Adjust the handlebar length using the release lever (Figure 26 Letter C). Once the handlebar height has been adjusted, make sure that the lever (Figure 26 Letter C) blocks the sliding movement and that is tightened securely (Figure 74).

Figure 74

	<p>Using the device with an incorrectly tightened lever (Figure 74) exposes the driver to the risk of losing control of the vehicle.</p>
--	--

11.2 HOW TO REMOVE THE HANDLEBAR WHEN THE TWIST IS STILL INSTALLED TO THE WHEELCHAIR

	<p>Make sure the Handlebar Control is switched off before disconnecting the handlebar from the TWIST. An unintentional press on the accelerator would cause the TWIST to move.</p>
	<p>Always perform this procedure with the brakes of the wheelchair engaged.</p>
	<p>Make sure there are no other controls switched on when switching off the Handlebar Control.</p>

- After ensuring that the Handlebar Control is switched off, completely unscrew the fixing knob by turning it anti-clockwise (Figure 75).

Figure 75

- Raise the handlebar upwards and perpendicular to the steering system of the TWIST (Figure 76).


Figure 76

11.3 LINKING THE TWIST AT THE FRONT WITH THE HANDLEBAR ALREADY INSTALLED

Figure 77

	<p>It is very dangerous to perform the operation of mounting the handlebar to the TWIST if the control is on. Make sure the Handlebar Control is deactivated.</p>
	<p>Always carry out the linking operation on a flat surface and if possible with the wheels of the wheelchair and of the TWIST lying on a smooth surface.</p>
	<p>Follow the instructions carefully to prevent the handlebar from hitting the user violently during the linking operations.</p>
	<p>When linking the TWIST, the handlebar must always be held with one hand.</p>
	<p>The TWIST with the handlebar engaged may under no circumstances be installed and uninstalled using only one hand. Always use two hands, one of which must always hold the handlebar.</p>
	<p>Always switch off the Handlebar Control before installing the handlebar and/or TWIST to the wheelchair.</p>
	<p>The Handlebar Control must never be switched on if the handlebar and TWIST are not installed to the wheelchair.</p>
	<p>It is advisable to avoid using the device when the Handlebar Control's battery is running low as, if its battery runs out, the TWIST will perform a safety stop and can then only be moved by manual push (no function is active when the Controller is switched off).</p>
	<p>When mounting the handlebar and when connecting the Handlebar Control to the TWIST device, make sure that no other controls in the immediate surroundings are switched on.</p>

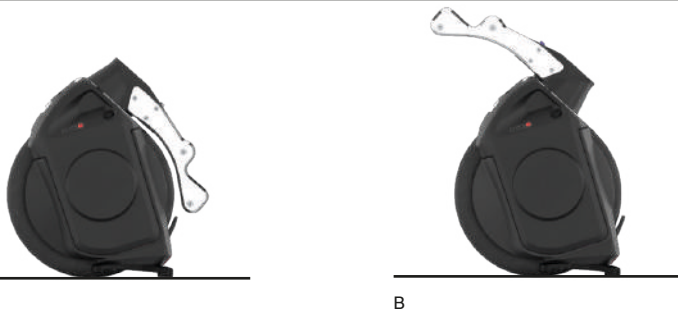A

B

Figure 78

- Place the TWIST on its kickstand and on a flat surface (Figure 78 Letter A).
- Rotate the steering system by 180° so that the upper and lower pin bracket (Figure 1 Letter G) points towards the front light (Figure 78 Letter B).
- Ensure that all controls are switched off. Hold the handlebar, making sure that the two alignment arrows are aligned with each other. Only in this position will it be possible to mount the handlebar (Figure 79).

Figure 79

Do not use the device with the handlebar if the arrows in Figure 79 are not aligned. In case the arrows are not aligned, the controls are not in the expected position for driving straight.

- Insert the handlebar base into the steering base (Multi-link Klaxon®) with a movement that is perpendicular to the steering of the TWIST (Figure 80 Letter A).
- Make sure that the handlebar is fully inserted and fully in contact with the steering base. (Figure 80 Letter B).

A

B

Figure 80

- Screw the knob clockwise until it is as tight as possible. Caution: do not use tools, but only the hands (Figure 81).

Figure 81

Using the device with the knob in Figure 81 not securely tightened exposes to the risk that the handlebar disengages with a consequent loss of control of the device.

- Press the red button (Figure 60 Letter A) on the side of the Front Linking System 2.0 to make sure that the hook opening is free and ready to accommodate the upper pin. If the button is locked, the system is ready to be linked (Figure 60).
- Grab the handlebar and tilt the TWIST towards the wheelchair to position the upper pin (in orange in Figure 83) inside the hook opening (Figure 82) and (Figure 84).

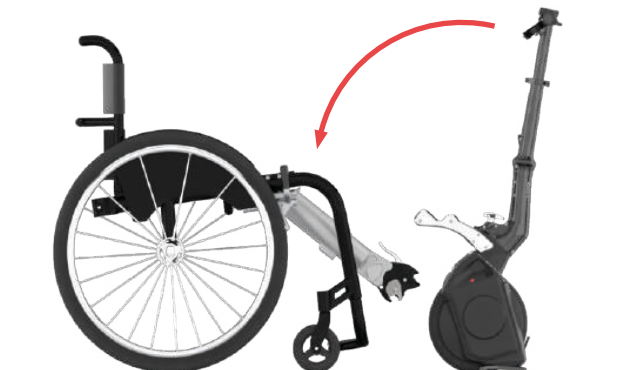
Figure 82

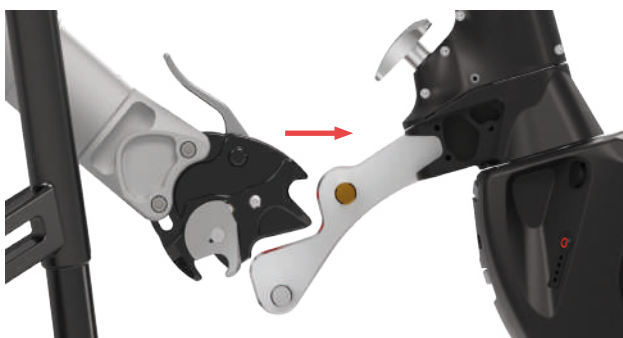
Figure 83

- Lift the lever (Figure 60 Letter B) to engage the hook and secure the TWIST. Ensure that the red button moves outwards until the green ring on the left appears. The green ring confirms the correct engagement (Figure 84). If the green mark does not appear, repeat the operation from the start.

Figure 84

If the green mark does not appear when the operation is repeated, this means that the Front Hook 2.0 Klaxon® is not properly engaged. Please contact a KLAXON service centre to carry out a technical check.

- Make sure that the wheelchair brakes are not applied (Figure 85).

Figure 85

- Push the handlebar forward until the lower pin is engaged and the wheelchair's front castor wheels are lifted off the ground (Figure 86).

Figure 86

- Turn the handlebar 180° to have the handlebar correctly facing the wheelchair and thus also be ready for driving forward (Figure 87).

Figure 87

Once the device has been installed to the wheelchair, it is important to check that the front castors of the wheelchair are between 25 mm and 40 mm above the ground. If this requirement is not fulfilled, do not use the device under any circumstances and contact an authorised KLAXON service centre immediately.

11.4 UNLINK THE TWIST FROM THE WHEELCHAIR WHILE KEEPING THE HANDLEBAR INSTALLED

	It is very dangerous to perform the operation of removing the handlebar from the TWIST if the control is on. Make sure the Handlebar Control is deactivated.
	Make sure there are no other controls switched on when switching off the Handlebar Control.
	Follow the instructions carefully to prevent the handlebar from hitting the user violently during the unlinking operations.
	When unlinking the TWIST, the handlebar must always be held with one hand.
	The TWIST with the handlebar engaged may under no circumstances be installed and uninstalled using only one hand. Always use two hands, one of which must always hold the handlebar.
	Always carry out the unlinking operation on a flat surface and if possible with the wheels of the wheelchair and of the TWIST lying on a smooth surface.

- Stand with the wheelchair on a flat and smooth surface on which the wheelchair can stop without having to engage the wheelchair brakes. Caution: with the wheelchair brakes engaged, the operation cannot be performed.
- Make sure the Handlebar Control is switched off.
- Turn the handlebar and then the TWIST by 180° until the front light of the TWIST is facing the wheelchair (Figure 88).

Figure 88

- Make sure the wheelchair brakes are not engaged.
- With one hand push the handlebar forward with the palm placed in the centre of the handlebar and with the other hand lift the lever (Figure 60 Letter B) on the Front Hook 2.0 Klaxon®. At this point the weight of the user will release the Front Hook 2.0 Klaxon® from the lower pin and the wheels of the wheelchair will lower to the ground (Figure 89). Accompany the lowering with the hand so that the TWIST does not drop lower abruptly.

	This operation must be carried out only by a user with the complete control over both its hands and who must have sufficient motor skills to perform the action described.
	There is a risk of injury if the handlebar descent is not accompanied correctly during the unlinking phase.

Figure 89

- While holding the handlebar firmly with one hand or, in alternative, while placing the handlebar on the user's legs, press the red button on the left side of the Front Hook 2.0 Klaxon® to disengage the upper hook and unlink the TWIST (Figure 90).

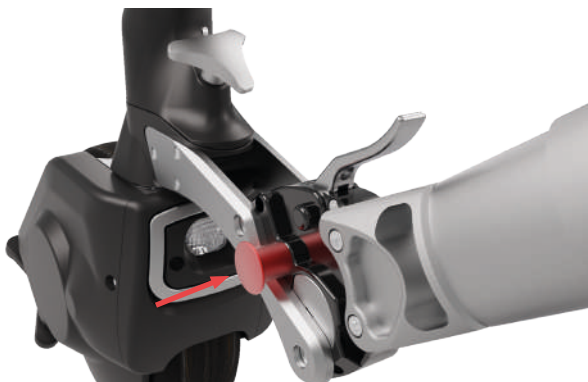
Figure 90

- Raise the handlebar to a vertical position and park the TWIST with the handlebar installed. The TWIST, placed in this way, is already in the ideal position to be installed again (Figure 91).

Figure 91

12. INSTALLATION OF THE TWIST AT THE REAR OF THE WHEELCHAIR

When linking the TWIST at the rear, it is possible to decide whether to link it by the manual procedure or by adopting the automatic procedure described as follows. Before carrying out the automatic procedure, it is mandatory to read the warnings very carefully.

Preparing the TWIST for linking

Installation of the specific frame (Figure 48 Letter D) on the TWIST

NOTE	Before attaching the frame to the TWIST, ensure that the battery is present and that it is inserted correctly and all the way into its housing.
NOTE	It will not be possible to insert the battery after installing the frame.

- Place the TWIST on a flat terrain (Figure 92).

Figure 92

- Insert the groove of the frame on the lower pin (Figure 93).

Figure 93

- Push the frame all the way through into position (Figure 94).

Figure 94

- Screw in the fixing knob (Figure 95).

Using the device with the knob in Figure 95 not securely tightened exposes to the risk of the frame disengaging and to the consequent loss of control of the device.

Figure 95

- Park the TWIST on its new front kickstand (Figure 48 Letter F). (Figure 96)

Figure 96

12.1 MANUAL LINKING OF THE TWIST TO THE REAR OF THE WHEELCHAIR

	<p>It is very dangerous to carry out the linking operation of the TWIST to the wheelchair if the Controller is switched on. Make sure the Controller is switched off before proceeding. Switch on the Controller only when the TWIST linking procedure has been completed.</p>
	<p>Always carry out the linking operation on a flat surface and, if possible, with the wheels of the wheelchair and the rubbers of the TWIST front kickstand resting on a smooth surface.</p>
	<p>The Controller must never be switched on if it is not connected to the wheelchair.</p>
	<p>Once the TWIST is installed to the wheelchair, make sure that only the Controller is activated. Make sure that any other controls are switched off.</p>
	<p>It is advisable to avoid using the device when the Controller's battery is running low as, if its battery runs out, the TWIST will perform a safety stop and can then only be moved by manual push (no function is active when the Controller is switched off).</p>
	<p>Always perform this procedure with the brakes of the wheelchair engaged.</p>

- Turn on the TWIST by pressing the button on the battery.
- Grab the TWIST from the handle (Figure 97).

Figure 97

- Place the TWIST on its front kickstand (Figure 48 Letter F). (Figure 98)

Figure 98

- Grip the handle and bring the TWIST closer to the hook until the lower pin rests perfectly against the hook (Figures 99 and 100).

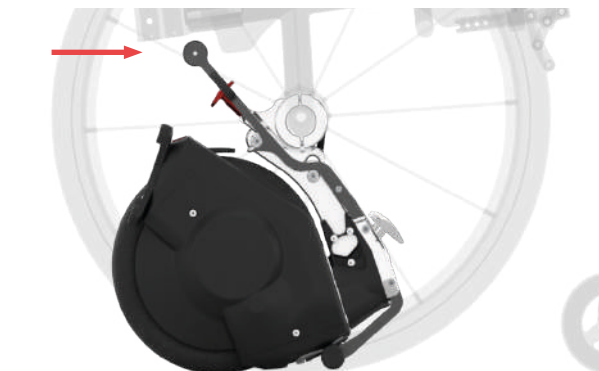
Figure 99

Figure 100

- Push the handle forward and simultaneously rotate the TWIST to link it. You will hear the typical "click" that confirms that the TWIST has been installed to the wheelchair. Move the handle up and down with force to check that it has been correctly engaged (Figure 101).

Figure 101

Using the device when not properly installed exposes the driver to the risk of losing control of the device.

12.2 AUTOMATIC LINKING OF THE TWIST TO THE REAR OF THE WHEELCHAIR

	The Controller must never be switched on if it is not connected to the wheelchair.
	Once the TWIST is installed to the wheelchair, make sure that only the Controller is activated. Make sure that any other controls are switched off.
	It is advisable to avoid using the device when the Controller's battery is running low as, if its battery runs out, the TWIST will perform a safety stop and can then only be moved by manual push (no function is active when the Controller is switched off).
	Always carry out the linking operation on a flat surface and, if possible, with the wheels of the wheelchair and the rubbers of the TWIST front kickstand resting on a smooth surface.
	It is very important to read the entire sequence described before performing this procedure .
	Before performing this procedure, it is very important that the manual procedure explained in section 12.1 has been carried out correctly at least one time.
	Do not perform this operation if there are obstacles of any kind in front of the wheelchair, be they things, people or animals.
	Always perform this procedure with the brakes of the wheelchair engaged.
	Perform this operation for the first time only in the presence of a support person. Continue to perform this operation with a support person until the technique has been mastered.
	We recommend performing this procedure only if a rear anti-tip system is installed on the wheelchair.

- Place the TWIST in front of the hook. The upper pin should rest in the opening of the hook located on the wheelchair (Figure 102).

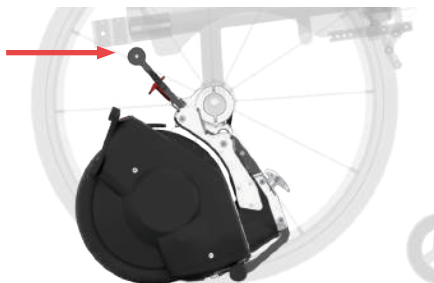
Figure 102

- Make sure there are no obstacles in front of the wheelchair for at least a few metres.
- Using the Controller, engage the first gear (see section 9.2). The TWIST will automatically link by moving upwards. At this point stop the power supply to the TWIST using the stop button on the Controller.
- The system is installed and ready for use. To start moving, release the wheelchair brakes and engage the gear.

	Using the device when not properly installed exposes the driver to the risk of losing control of the vehicle.
---	---

12.3 UNLINKING OF THE TWIST WHEN INSTALLED TO THE REAR OF THE WHEELCHAIR

	<p>Always carry out the unlinking operation on a flat surface and, if possible, with the wheels of the wheelchair and the rubbers of the TWIST front kickstand resting on a smooth surface.</p>
	<p>It is very dangerous to unlink the TWIST from the wheelchair if the Controller is switched on. Make sure the Controller is switched off before proceeding with the unlinking operation.</p>
	<p>Make sure there are no other controls switched on when switching off the Controller.</p>
	<p>When releasing the TWIST, it is recommended to lock the wheels of the wheelchair using the brakes provided with the wheelchair.</p>

- Make sure that the Controller is switched off and thus that all LEDs are off.
- Place the palm of the hand on the frame's handle and pull the release/engagement lever (A) with the fingers. (Figure 103)
- The TWIST is now uninstalled and can be removed by grabbing it by the handle of the frame and pulling it back (Figure 104).

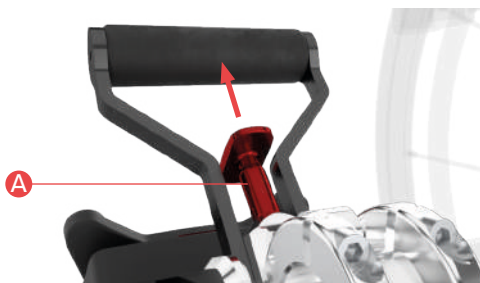
Figure 103

- Place the TWIST on its front kickstand (Figure 105).

Figure 104

- Switch off the TWIST by pressing the off button on the battery.

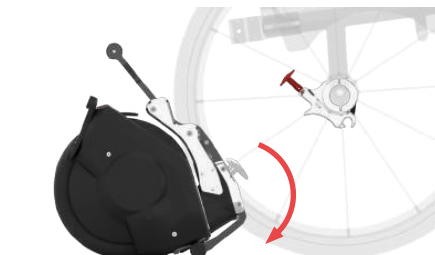
Figure 105

13. IMPORTANT NOTES AND PRECAUTIONS CONCERNING THE USE OF THE TWIST

Where and how to use the TWIST

In all its configurations, the TWIST is designed for both indoor and outdoor use, but not for off-road use. It is, however, possible to negotiate short off-road routes as long as the terrain is dry, whether it is paved or on grass. On the other hand, it is not possible to ride on terrain with significant unevenness, mud or puddles of water.

	Do not use the TWIST on off-road routes with major irregularities.
	Do not use the TWIST on sandy and muddy off-road tracks.
	Do not drive the TWIST over puddles with a water level higher than the height of the tyre itself. The maximum permitted height is up to the lower part of the wheel casing on the TWIST (Figure 106).
	In off-road use it is recommended to always use the handlebar accessory installed.

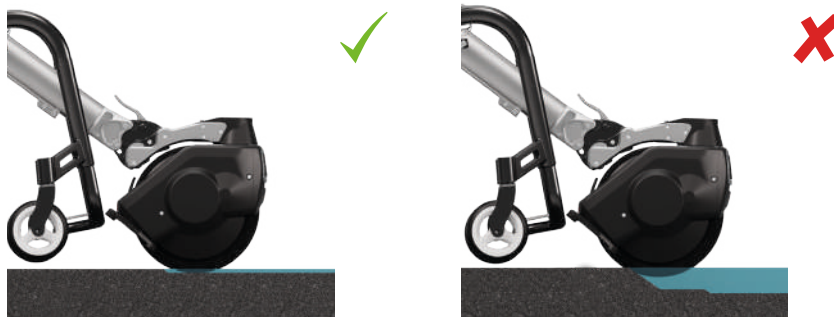
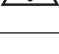
Figure 106

	Always drive over puddles at low speed, taking care that the water level does not exceed the one indicated in the previous point. By low speed is meant the lowest possible speed.
	The electronic brake only works when the TWIST is installed at the front with or without the handlebar equipped.
	The mechanical brake only works when the TWIST is installed at the front and with the handlebar equipped.
	When the TWIST is installed at the rear, the wheelchair cannot be slowed down or braked using the Controller. When the TWIST is rear-installed, braking must be performed by acting on the manual wheelchair by actively blocking its movement. It is also necessary to act on the manual wheelchair to keep it stationary in position.
	The TWIST installed at the rear does not allow the front castor wheels of the wheelchair to be lifted mechanically off the ground. For this reason, the use of the rear-installed TWIST is recommended on terrain with no bumps or small steps that could block the front wheels of the wheelchair, causing the wheelchair to catapult the passenger dangerously forward. For use on uneven terrain and therefore generally in the outdoors, link the TWIST at the front (with or without handlebars).

KLAXON Mobility GmbH - EN

	When using the TWIST rear-installed, the maximum step that can be negotiated is the one declared by the wheelchair manufacturer. Always switch off the power supply to the motor when negotiating a step and switch to manual driving mode.
	When the device is installed at the front, the maximum step that can be negotiated is 30 mm.

Figure 107

	The TWIST may only be used on routes with inclinations of no more than 6° / 10%.
	At high speeds the user may lose control of the wheelchair and a tip over may occur. Never exceed the maximum speed of 20 km/h in handlebar mode (6 km/h in Controller mode) when going uphill or downhill. Avoid collisions in general.
NOTE	Do not lean the TWIST against steps or obstacles higher than 120 mm, as this could damage the plastic chassis. Below 120 mm it will be the tyre that will safely lean against the obstacle (Figure 108).

Figure 108

	The motor of the TWIST device does not perform any braking and/or blocking action when the device is not in motion (recorded speed 0 Km/h). This also refers to the braking action performed by the Electronic Brake System (EBS) when the device stops (reaches 0 km/h). It is therefore necessary to act on the manual wheelchair by actively blocking its movement. It is also necessary to act on the manual wheelchair to keep it stationary in position. If the handlebar is used, it is also possible to act on the mechanical and parking brake of the TWIST (provided that the handlebar is properly connected to the device) to block the movement and keep the wheelchair locked in position.
---	--

14. DRIVING THE TWIST

Depending on the installation position of the TWIST and which accessory will be used, the driving style of the TWIST changes radically. Below you can find the necessary instructions for an easy and safe driving in all the different configurations. The TWIST guide can also be customised using the KLAXON Web App accessory, which can be downloaded and used at any time.

14.1 MANAGEMENT (ON/OFF) OF THE STEERING DIRECTIONALITY CONTROL SYSTEM

The TWIST is equipped with a mechanical system that by acting on the steering, facilitates the steering directionality while driving (Figure 109).

Figure 109

Thanks to this solution, the TWIST is able to maintain the directionality more easily when riding uphill and when going against gradient. To activate or deactivate the system, simply turn the adjustment screw with a coin. The small dot at the top indicates that the steering control system is active. When the dot is located on the bottom it indicates that the system is switched off (Figure 110).

Figure 110

The dot on the lower part indicates that the system is deactivated and therefore the TWIST will not assist in maintaining the directionality

The dot on the upper part indicates that the system is activated and therefore the TWIST will assist in maintaining directionality

14.2 DRIVING THE TWIST WITH THE CONTROLLER WHEN INSTALLED AT THE FRONT

The front-installed TWIST allows the front castors of the wheelchair to be raised off the ground. This allows to move around safely even on uneven terrain.

Before driving:

- Make sure the TWIST is installed to the wheelchair and that the wheelchair has its front castors lifted off the ground (see linking instructions in section 10.1).
- Make sure the TWIST is turned on by pressing the appropriate button on the battery. Two short acoustic signal will be emitted by the TWIST.
- Make sure the Controller is installed on the wheelchair.
- Make sure the Controller has been turned on by pressing the appropriate button on the control. The LEDs will light up and flash until the connection with the TWIST is made. A short acoustic signal will be emitted to confirm the successful connection, while the LEDs of the control will remain lit without flashing.

To start driving:

If the TWIST is installed at the front, it is possible to start and stop in two different ways.

	In either mode, the Controller must always be activated and connected to enable the device to brake.
---	--

Assisted start and braking through the use of the wheelchair's wheels

The motor can be activated, with the Controller active and connected, by pushing the wheelchair manually (Assisted Start). When the set speed for the 1st Gear (or one of the speed levels available in WebApp for this function) is reached, the motor will activate and maintain this speed. Once in motion, it is always possible to increase speed or stop the device via the Controller.

The device can also be braked manually by blocking the wheels of the wheelchair. This Assisted Braking function is always possible when the Controller is active and connected to the TWIST.

Start and stop through the use of the Controller

- To start, push the speed selection lever forward and release it to shift into gear 1. Repeat the same operation to increase speed. There are 4 speeds and the maximum speed will be 6 km/h.
- To reduce speed, press and release the speed selection lever in the opposite direction of the drive direction. With each press and release, the speed will be reduced until it reaches the zero gear, which corresponds to the neutral.
- To stop the device immediately, briefly press the stop button on the Controller.
- To slow down to 0 km/h and restart without remaining still, press and hold the stop button for more than 1.5 seconds. When released, the device will restart from the speed of the first gear.

Steering

The TWIST installed at the front, and without the handlebar accessory equipped, can be steered by braking and pushing the wheels of the wheelchair or by performing the two operations simultaneously to achieve a faster response, just as when steering the wheelchair without the TWIST installed. Please note that the power provided by the motor increases as the resistance increases (as, for example, when driving uphill) so as to maintain the set speed constant.

Example: to turn right, brake the right wheel and simultaneously push the left wheel. To turn left, brake the left wheel and simultaneously push the right wheel.

Automatic speed reduction when turning

In its front-installed configuration, the TWIST employs a number of electronic features to make the driving experience simple and intuitive. In particular, the automatic speed reduction when turning and the interruption of motor thrust when a turning angle of around 90°, to the left or right, is surpassed.

Once the curve has been completed and the wheel has been repositioned in the straight direction, the speed will increase to the speed that was set before the curve was executed (Figure 111).

Thanks to these electronic features, which can be modified and customised from the Web App, the driving experience will always be safe and adjusted to the user's needs and to its control capabilities.

Figure 111

Automatic maintenance of a set speed

Once a speed has been set via the Controller, this speed will be automatically maintained both uphill and downhill.

	<p>When driving downhill, the speed can be significantly higher than the set speed, according to the slope.</p>
---	---

Always-on driving safety

Safety is always guaranteed even if the control should disconnect or its battery runs out. In this case, the TWIST will stop immediately by braking automatically until it reaches 0 km/h.

	<p>The motor of the TWIST device does not perform any braking and/or blocking action when the device is not in motion (recorded speed 0 Km/h). This also refers to the emergency stop from disconnection when the device comes to a halt (reaches 0 Km/h). It is therefore necessary to act on the manual wheelchair by actively blocking its movement. It is also necessary to act on the manual wheelchair to keep it in stationary in position. If the handlebar is being used, it is also possible to act on the mechanical brake of the TWIST (provided that the handlebar is properly connected to the device) to block the movement and keep the wheelchair stationary in position.</p>
---	--

Monitoring driving information via smartphone

It is possible to use your smartphone as a dashboard to display driving parameters in real time. By attaching the smartphone to the wheelchair and activating the KLAXON Web App on the *Dashboard* page, it is possible to display the travel speed, the number of kilometres travelled, the battery charge level of the TWIST and of the connected control, and the remaining range of kilometres in real time.

	<p>When using the Web App accessory as a display, the smartphone or any other electronic device they must absolutely be attached to the wheelchair. Always use the TWIST device only with both hands free and available for driving.</p>
	<p>It is advisable to perform the first driving tests always and only on flat, obstacle-free terrain.</p>
	<p>Should the Controller be detached or become temporarily inaccessible to the driver, please remember that the Assisted Braking function is always available through the blocking of the wheelchair wheels. Alternatively, it will be sufficient to brake one of the two wheels of the wheelchair to steer and reduce speed until the motor's thrust is deactivated; this will occur when the steering angle exceeds 90°.</p>

CAUTION	The wireless control must always be firmly mounted to the wheelchair (see section 9.2).
---------	---

14.3 DRIVING THE TWIST WITH THE HANDLEBAR WHEN INSTALLED AT THE FRONT

For the installation of the handlebar please read chapter 11.

The front-installed TWIST with the handlebar accessory equipped allows the front castors of the wheelchair to be raised off the ground. For this reason this mode of use allows you to move around safely even on uneven terrain.

Before driving:

- Make sure all the controls are switched off.
- Make sure the TWIST is installed to the wheelchair and that the wheelchair has its front castors raised off the ground (see chapters 10 and 11).
- Make sure the TWIST is turned on by pressing the appropriate button on the battery. Two short acoustic signal will be emitted by the TWIST.
- Make sure the handlebar is securely mounted on the TWIST.
- Make sure the Handlebar Control has been turned on by pressing the appropriate button on the control. The LEDs will light up and flash until the connection with the TWIST is made. A short acoustic signal will be emitted to confirm the successful connection, while the LEDs of the control will remain lit without flashing.

Start driving and speed control

Use the accelerator to start driving and to set the desired speed within the maximum limit of 20 km/h. The accelerator can be configured by the dealer to brake the TWIST at the moment the lever is released (Motor Brake function). This function can be disabled completely. This driving option can be adjusted in intensity via the KLAXON Web App.

Adjusting the power delivery

The level of power delivery during acceleration can be adjusted via the Web app. A driving mode with smooth acceleration or one with a powerful acceleration can be set according to the user's personal driving needs.

Set the desired maximum speed

Using the Web app, it is possible to set the desired speed limit. The maximum speed that can be set is 20 km/h. The minimum speed that can be set is 6 km/h.

Proceed in reverse:

To proceed in reverse:

- Rotating the handlebar beyond 90 degrees makes the reverse function active and not enabled. An acoustic signal (Beep) will be emitted to indicate that the mode has been changed.
- Press the accelerator all the way down to the maximum speed position to enable the reverse gear. Release the accelerator. Press the accelerator gently to start the reverse motion. Please note that speed will be limited to 3 km/h.
- The device will automatically brake when the accelerator is released.

To return to the forward motion:

- Release the accelerator.
- Turn the handlebar to the forward drive position. An acoustic signal (beep) will be emitted to indicate the mode change.
- Proceed to drive forward normally.

Monitoring driving information via smartphone

It is possible to use your smartphone as a dashboard to display driving parameters in real time. By attaching the smartphone to the wheelchair and activating the KLAXON Web App on the *Dashboard* page, it is possible to display the travel speed, the number of kilometres travelled, the battery charge level of the TWIST and of the connected control, and the remaining range of kilometres in real time.

	When using the Web App accessory as a display, the smartphone or any other electronic device they must absolutely be attached to the wheelchair/handlebar. Always use the TWIST device only with both hands free and available for driving.
	Before unmounting the handlebar from the TWIST or before linking and unlinking the device, always ensure that the Handlebar Control is switched off.

In case of problems with the Handlebar Control connection

In the event that the Twist does not restart moving, stops or does not execute commands make sure to be in a location away from danger and follow the following steps: switch the Handlebar Control off and on again. If the problem persists: switch the Handlebar Control and the Twist off and on again. If the above steps do not solve the problem: Do not use the device and contact an authorised KLAXON service centre.

	Do not cover the Handlebar Control (Figure 17 Letter A) with the hand while driving. A hand placed on the Handlebar Control could reduce the range of the Bluetooth signal and lead to a possible disconnection from the device.
---	--

How to reduce the backlash between the Linking System and the device

It is possible, should it be considered necessary, to reduce the backlash between the device and the Linking System by tightening a knob screw (Figure 17 Letter K) supplied with the handlebar. Once the TWIST is installed, it is possible to reduce the backlash by screwing the knob screw by hand into its appropriate seat (Figure 1 Letter H) until it is completely tightened (Figure 112 Letter A). Subsequently, in order to unlink the TWIST device from the Linking System, it will be necessary to first unscrew the screw (Figure 112 Letter B) of a few turns, or until the system is free from the restriction.

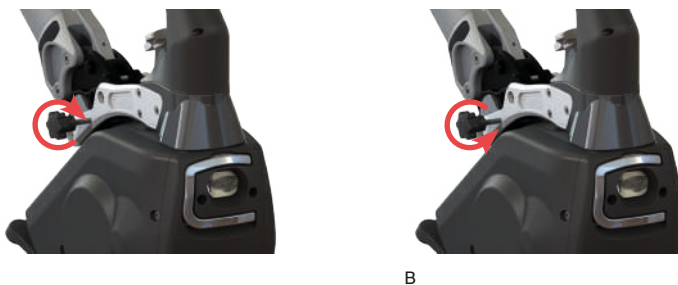
Figure 112

CAUTION	When linking and unlinking the device, always ensure that the knob screw does not obstruct the insertion of the Linking System. In this case, simply unscrew the knob screw a few turns until the Linking System engages without resistance.
	Make sure that the knob screw is tightened enough. If it remains loose, it could detach during the usage.

14.4 DRIVING WITH THE TWIST INSTALLED AT THE REAR

The TWIST installed at the rear does not allow the front castor wheels of the wheelchair to be lifted mechanically off the ground. For this reason, the use of the rear-installed TWIST is recommended on terrain with no bumps or small steps that could block the front wheels of the wheelchair, causing the wheelchair to catapult the passenger dangerously forward.

	In the rear-installed configuration, the TWIST does <u>NOT</u> use the electronic features that automatically reduce speed during curves as well as the automatic stop of the motor thrust when exceeding a steering angle of more than 90°. It is therefore recommended to reduce speed via the Controller before starting a curve.
---	--

	It is advisable to perform the first driving tests always and only on flat, obstacle-free terrain.
	The wireless control must always be firmly mounted to the wheelchair (see section 9.2).
	When using the TWIST installed at the rear, it is mandatory to have an anti-tipping system installed to the wheelchair. The anti-tipping system shall be extended and operative when using the TWIST device at the rear.
	In order to avoid interferences, make sure that the anti-tipping system is extended and operative before connecting the TWIST at the rear.

Before driving:

- Make sure the TWIST is turned on by pressing the appropriate button on the battery. Two short acoustic signal will be emitted by the TWIST.
- Make sure the TWIST is installed to the wheelchair (see chapter 12).
- Make sure the Controller is connected to the wheelchair.
- Make sure the Controller has been turned on by pressing the appropriate button on the control. The LEDs will light up and flash until the connection with the TWIST is made. A short acoustic signal will be emitted to confirm the successful connection, while the LEDs of the control will remain lit without flashing.

Start and stop through use of the wheelchairs' wheels

	When the TWIST is installed at the rear, the Assisted Start function is deactivated.
	When the TWIST is installed at the rear, it is not able to brake the wheelchair, in particular when driving downhill.

Start and stop through the use of the Controller

- To start, push the speed selection lever forward and release it to shift into gear 1. Repeat the same operation to increase speed. There are 4 speeds and the maximum speed is 6 km/h.
- To reduce speed, press and release the speed selection lever in the opposite direction of the drive direction. With each press and release, the speed will be reduced until it reaches the zero gear, which corresponds to the neutral
- To deactivate the motor thrust, briefly press the Controller's stop button.
- To temporarily deactivate the motor thrust and then reactivate it, press and hold the button for longer than 1.5 seconds. When released, the device will restart from the speed of the first gear.

Steering

The TWIST installed at the rear can be steered by braking and pushing the wheels of the wheelchair, or (recommended) by performing both operations simultaneously, just as when driving the wheelchair without the TWIST installed. Please note that the power provided by the motor increases as the resistance increases (as, for example, when driving uphill) so as to maintain the set speed constant.

An example: to turn right, brake the right wheel and push the left wheel at the same time. To turn left, brake the left wheel and at the same time push the right wheel. It is advisable, particularly in the initial learning stages, to reduce the speed before commencing a curve by acting on the Controller (see section 9.2) (Figure 113).

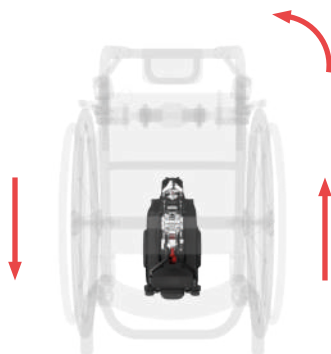
Figure 113

Brake

When the TWIST is rear-installed, the only braking action possible is by acting on the wheelchair by stopping the handrails or, in the case of a third person controlling it, by stopping the wheelchair. In both cases the wheelchair must undergo a drastic reduction in speed that brings it close to 0 km/h. The Controller's stop button will interrupt the power supply from the motor, but the motor will not be able to slow down the device, in particular when travelling downhill.

Always-on driving safety

Safety is always guaranteed even if the control should disconnect or its battery runs out. In this case, the TWIST will stop immediately.

The motor stop also comes into operation in case the control detaches from the wheelchair. In this case, once a certain distance has been passed, approximately 10 metres, the motor of the TWIST will stop automatically.

The motor of the TWIST device does not perform any braking and/or blocking action when the device is not in motion (recorded speed 0 Km/h). This also refers to the emergency stop from disconnection when the device comes to a halt (reaches 0 Km/h). It is therefore necessary to act on the manual wheelchair by actively blocking its movement. It is also necessary to act on the manual wheelchair to keep it stationary in position.

Monitoring driving information via smartphone

It is possible to use your smartphone as a dashboard to display driving parameters in real time. By attaching the smartphone to the wheelchair and activating the KLAXON Web App on the *Dashboard* page, it is possible to display the travel speed, the number of kilometres travelled, the battery charge level of the TWIST and of the connected control, and the remaining range of kilometres in real time.

When using the Web App accessory as a display, the smartphone or any other electronic device they must absolutely be attached to the wheelchair. Always use the TWIST device only with both hands free and available for driving.

14.5 USE OF THE TWIST DEVICE BY A THIRD PERSON

Should the TWIST be operated by a third person, the latter must read and fully understand this User Manual, the manual of all the accessories and of the wheelchair itself. We also recommend reading sections 14.2 thoroughly if the TWIST is front-installed, or 14.4 if it is rear-installed. Please also read thoroughly section 9.2 concerning the use of the Controller.

Linking the TWIST at the front will facilitate driving over rough terrain and also allows the electronic braking management on both flat and downhill terrain.

Linking the TWIST at the rear will promote a greater freedom of movement on flat terrain and particularly in indoor environments. In this configuration, the motor brake management will not be active both when riding on flat ground or downhill.

When installed at the rear, pay the utmost attention while driving to any obstacles that could suddenly block the front castors of the wheelchair and, as a consequence, catapult the seated person forward.

When a third person is driving, the Controller must be installed on the rear handles of the wheelchair.

Never leave the wheelchair unattended with the Controller installed and active. An accidental pressure of the control could make the device move.

15. INSTRUCTIONS FOR MAINTENANCE AND SERVICING

15.1 CHARGING THE TWIST BATTERY

Figure 114

- Switch off the TWIST by pressing the button on the battery.
- Connect the battery charger to the domestic power socket.
- Lift the rubber cap protecting the battery terminal and connect the plug to the battery of the TWIST.
- The battery LEDs will light up in sequence and the LED on the charger will glow red until the battery is fully charged.
- When the battery is fully charged, the battery LEDs will all remain lit without flashing and the charger LED will glow green.
- Disconnect the battery charger from the wall socket and from the battery and put the protective cap on the connector back on.
- To ensure that the charge has been completed, press the battery button, turn it on and check that all 4 LEDs are lit.

15.2 CHARGING THE CONTROLLER OR THE HANDLEBAR CONTROL'S BATTERY

Figure 115

- Switch the control off by pressing the button.
- Connect the battery charger to the domestic power socket.
- Connect the mini USB connector to the control.

- The LEDs of the control will light up indicating the charge level of the control.
- When the battery is fully charged, the battery LEDs will all switch off.
- Disconnect the charger from the wall socket and from the control.
- To ensure that the charge has been completed, press the control button by switching it on and check that all 4 LEDs are all lit.

15.3 TYRE INFLATION

TWIST's tyre is one of those with an inner tube inside. The tyre inflation pressure must vary between 2 bar (30 psi) and 2.5 bar (36 psi). Inflation can also be carried out by the user or at a motorbike or bicycle workshop or at a car tyre shop.

- To access the inflation valve, pull out the battery. A hole will be visible at the bottom. This hole will provide direct access to the wheel rim (Figure 116).

Figure 116

- Turn the wheel until the valve appears.
- Unscrew the valve cap.
- Turn the valve slightly so that the head of the inflation gun's hose can be inserted into it.
- Check that the pressure is between 2 bar (30psi) and 2.5 bar (36psi).
- Screw the plastic cap back on.
- Rotate/push the valve until it rests on the rim.

Make sure that when turning the wheel the valve does not touch the plastic chassis.

- Reinsert the battery.

15.4 TYRE AND INNER TUBE REPLACEMENT

If the inner tube is punctured or the tyre needs to be replaced because it is too worn, please contact your KLAXON dealer or service centre.

15.5 REPLACEMENT OF COMPONENTS

For the replacement of the mechanical brake cable on the handlebar accessory, for the replacement of the mechanical brake cable on the TWIST or of the brake pads, please contact your dealer or a KLAXON service centre.

16. INSTALLATION

The TWIST device is provided by a KLAXON-authorized technician only after the Linking System has been pre-installed on the wheelchair.

	<p>The TWIST device and the Linking Systems can only be installed on compatible manual wheelchairs. KLAXON provides its distributors with an Installation Manual that contains the linking criteria. KLAXON products and Linking Systems may only be installed by authorised KLAXON technicians who can decide on their own responsibility whether the specific manual wheelchair can be used properly with the TWIST device and the chosen Linking System (based on both the characteristics of the manual wheelchair and the overall conditions).</p>
---	---

17. MAINTENANCE

The TWIST device, its components, the Linking Systems and accessories must be serviced regularly, only in this way can the safety, reliability and effectiveness of the device be guaranteed.

	<p>KLAXON is not liable for any damage caused by a lack of or for an insufficient maintenance. In such cases the validity of the warranty will be voided. The use of the device that has not undergone the maintenance procedures is considered as improper and thus effectively voids the warranty. The user is fully and solely responsible for any injury or damage resulting from a malfunction of the device due to not performing the mandatory maintenance.</p>
	<p>The TWIST device may only be repaired by authorised KLAXON technicians. Unscheduled maintenance can be carried out only by authorised KLAXON technicians.</p>
	<p>It is forbidden to remove any components from the TWIST device. Only authorised KLAXON technicians may remove components from the TWIST device. The only component that can be removed by the user is the battery.</p>
	<p>It is mandatory to verify the correct installation after one month. In order to do so please contact an authorised KLAXON technician. This operation will check the system settlement.</p>
<p>NOTE</p>	<p>Please contact a Klaxon Dealer to know whether a spare part is available or not.</p>

17.1 PERIODICAL MAINTENANCE

The following periodical maintenance activities are mandatory:

	<p>Check the tyre inflation pressure at least once a month; a deflated tyre compromises the smoothness of the ride, the safety and the electric consumption. The TWIST tyre must be inflated between 2 bar (30 psi) and 2.5 bar (36 psi).</p>
	<p>Check the wear condition of the tyres at least once a month. Contact an authorised KLAXON technician to replace the inner tube or tyre.</p>
	<p>Check, at least once every two months, the tightening of all screws and fasteners of the device and of the Linking Systems.</p>
	<p>Check the tightening of the wheel axle bolts at least once a year with a dynamometric torque wrench set to 25 Nm. Contact an authorised KLAXON technician.</p>

	Before each use, check that the accelerator and brakes are working properly; any replacement of wear parts (brake pads, discs, braking system in general) should only be carried out by authorised KLAXON technicians. It is also recommended to have the device checked whenever it is perceived that the braking capacity has diminished.
	Check that the battery is in good condition before each use. For the replacement of the battery, and any other electrical components, please contact only authorised KLAXON technician.
	Be careful when using high pressure air to inflate the tyre.
	Have a KLAXON authorized technician overhaul the entire device at least once a year, and/or whenever a malfunction occurs.
	The device must be checked by a KLAXON's authorized technician whenever you feel that the braking power has decreased.

17.2 MALFUNCTIONS

Should a malfunction occur when using the TWIST device, its components, Linking Systems and accessories, please contact KLAXON or an authorised KLAXON dealer.

Malfunction found	Possible cause	Corrective action
The device does not turn on	The battery is empty	Recharge the battery.
	The battery is positioned improperly	Check the position of the battery by removing it and inserting it again, making sure that the electrodes are inserted in the appropriate spaces.
	Generic problems	Do not use the device. Contact an authorised KLAXON service centre.
It vibrates excessively during use	Loose screws	Do not use the device. Contact an authorised KLAXON service centre.
The TWIST is not manoeuvrable	The front tyre is flat	Inflate tyre.
	The front tyre is punctured	Do not use the device. Contact an authorised KLAXON Dealer.
	The mounting angle of the device is incorrect	Do not use the device. Contact an authorised KLAXON Dealer.
The controls switch on but the device does not respond	The battery is empty	Recharge the battery.
	Pairing was not performed or not performed correctly	Contact an authorised KLAXON Dealer.
	Generic problems	Do not use the device. Contact an authorised KLAXON Dealer.
The front castors of the wheelchair touch the ground	The frontal Linking System is installed too low	Do not use the device. Contact an authorised KLAXON Dealer.
The Linking System does not engage	Presence of dirt on the Linking System	Clean the Linking System of all dirt and then apply a lubricant.

Malfunction found	Possible cause	Corrective action
TWIST does not respond to commands	The control is switched off or its battery is out of charge	<p>If in motion: stop the device and the wheelchair. Move to a safe position away from danger. Switch on or recharge the control.</p> <p>If not in motion: move to a safe position away from danger. Switch on or recharge the control.</p>
	Generic problems	Do not use the device. Contact an authorised KLAXON Dealer.
Twist does not restart moving, stops or does not execute commands	Command connection problems	<p>Make sure to be in a position away from danger and carry out the following steps:</p> <p>1) Press the stop button (if in Controller mode). If the problem still persists:</p> <p>2) Switch the comand in use off and on again. If the problem still persists:</p> <p>3) Switch the comand in use and the Twist off and on again.</p> <p>If this does not solve the problem: Do not use the device. Contact an authorised KLAXON Dealer.</p>

	<p>Never use the TWIST device if the traction unit, the accessories and/or the Linking System are behaving abnormally. Immediately contact a Klaxon Dealer.</p>
---	---

17.3 SERVICE LIFE OF THE PRODUCT

After careful analysis of the market, the technologies used, the state of the art, the intended use and the maintenance specifications KLAXON has estimated that the product has a service life of approximately 5 years. KLAXON would like to emphasize, in a very clear way, that this period is still an estimation and that it can be extended beyond this date if the user follows the correct maintenance procedures, uses the product properly and with the appropriate care.

If the TWIST device reaches the end of this period please contact your health insurance, local health service, KLAXON dealer or the shop where the product was purchased.

18. BATTERY INFORMATION

18.1 TRANSPORTATION, STORING AND GENERAL INFORMATION

- The batteries supplied with the TWIST device are compliant with the EMC2004/108/EC regulation issued by the European Union regarding the CE Classification. It is therefore permitted to transport these batteries on a train, ship or airplane. Despite of this classification they are considered, according to the UN Recommendations on the Transport of Dangerous Goods, as dangerous material and classified as Class 9, respectively as: code UN3480 (Lithium Ion Batteries) if transported as single units and UN3481 (Lithium Ion Batteries Contained in Equipment) if transported while connected to the TWIST device. This classification determines some restrictions regarding the transportation of the product by ship (IMDG Code) and by plane (IATA DGR) referring to: quantity of batteries transported, type of transport (if transported as single unit or connected to the device) and the Watt-hour value (Wh) of each single battery. In this regard, if you are planning to take your device with you, please contact directly the shipping company or the airline chosen for your travel.

Here below a list with the details of the available battery models supplied by KLAXON:

– TWIST battery: 36V x 4Ah (144 Wh)

- We recommend paying attention to the charge level indicator! Always recharge when the level is low. We always advise you to recharge the battery after extended use. Lithium-ion batteries have no memory effect, so they can be charged without having to wait until they are completely discharged.
- Store the battery charger away from heat sources, in a dry place and protected from direct sunlight.

	When not using the battery, remember to charge it at least once a month.
	If you see that the battery is damaged, its housing is broken, if it swells or leaks, absolutely do not use it and contact KLAXON's Technical Support immediately.
	Only use the supplied battery charger together with the TWIST device: any damage or malfunction due to non-compliance with these instructions or use of products that do not comply with KLAXON's guidelines will not be covered by the warranty.
	There is a risk of injury due to short-circuits and risk of electric shock if the battery charger has been damaged. Avoid using the battery charger if it has been dropped or damaged.
	Avoid the use of extension cords unless absolutely necessary. When using them, make sure in advance that they are undamaged and in good condition to avoid risks of fire and electrical shock.
	Once the battery is fully charged, the charger automatically cuts the power supply. Do not leave the battery charger connected to the 230 V mains for too long after it has finished its charge cycle (green LED on).
	Always charge the battery in a well ventilated room and keep it away from flammable material.
	Always charge the battery before you start using the TWIST. Never use the device with an empty battery.

19. TECHNICAL DATA

19.1 CLASSIFICATIONS OF THE DEVICE

According to the Medical Devices Directive Annex VIII EU745/2017 MDR	Risk class I
Protection against electrical hazards	Class II
Degree of protection against direct and indirect contact	Type B
Use in oxygen-rich environments	No protection
Operating conditions	Device for continuous operation

19.2 PERFORMANCE

#	Feature	Value	Unit	Notes
1a	Available Speeds - Usage with Controller	2 - 3.5 - 4.5 - 6	Km/h	Other maximum speeds values depend on national regulations.
1b	Available Speeds - Usage with handlebar	from 0 to 20	Km/h	Other maximum speeds values depend from customised settings or national regulations.
2	Range of the TWIST battery	20	Km	With a fully charged battery, at moderate speed, on flat ground and with a user weighing 70 kg
3	Maximum negotiable rated slope	6	°	With a fully charged battery and user weighing 70 kg. The 6° corresponds to a 10% slope.
4	Maximum negotiable obstacle height	30	mm	With a fully charged battery and user weighing 70 kg
5	Maximum negotiable ground unevenness height	30	mm	With a fully charged battery and user weighing 70 kg

KLAXON Mobility GmbH - EN

6	Motor voltage	36	V	
7	Maximum weight of user	120	Kg	Consistently with the technical characteristics of the wheelchair to which it is installed to.
8	Expected service life	5	years	Approximately 25,000 km.

19.3 DIMENSIONAL DATA

#	Feature	Value	Unit	Notes
1	Length	310	mm	
	Length with handlebar	420	mm	
2	Width	150	mm	
	Width with handlebar	450	mm	
3	Height	340	mm	
	Height with handlebar	940	mm	
4	Weight	7.4	kg	without battery.
		9.8	kg	with handlebar, without battery.
5	Motor power	250	W	750 W of peak power.
7	Tyre Size	10	inch	70/65 6.5

19.4 ENVIRONMENTAL OPERATING CONDITIONS

#	Feature	Value	Unit	Notes
1	Maximum altitude	2,000	m	
2	Atmospheric pressure	700 to 1,060	hPa	
3	Temperature	-25 to 50	°C	0°C to 40°C is suggested for a better performance.
4	Relative humidity	30 to 90	%	

19.5 TRANSPORTATION AND STORAGE ENVIRONMENTAL CONDITIONS

#	Feature	Value	Unit	Notes
1	Temperature	-40 to 65	°C	10°C to 50°C is suggested.
2	Relative humidity	30 to 90	%	

NOTE	<p>Before a long period of storage, charge the battery between 30% and 80%.</p> <p>During a long period of storage, check the battery's charge level periodically and charge it at least once a month until it returns to the aforementioned values.</p> <p>Before starting to use the battery again after a long period of storage, fully charge the battery.</p>
------	--

19.6 TECHNICAL DESCRIPTION OF BATTERY CHARGER

#	Feature	Value	Unit	Notes
1	Input voltage	230	V	
2	Frequency	50	Hz	
3	Output Voltage	48	V	

19.7 TECHNICAL DESCRIPTION OF THE BATTERY

NOTE	All KLAXON products are provided with a Li-Ion Maintenance Free battery.
------	--

A. TWIST BATTERY (144 Wh)

#	Feature	Value	Unit	Notes
1	Rated voltage	36	V	
2	Capacity	4	Ah	
3	Charging time	approx. 2	h	Time to charge a completely discharged battery.

19.8 MAIN FEATURES

#	Feature	Value
1	Traction type	Electric
2	Brake	Disc (diameter 150 mm), mechanical
3	Type of use	Outdoor and indoor

20. CLEANING

A regular cleaning of the device, in all its parts, guarantees a longer life and better functionality of the device.

It is recommended to (except as provided for in section 20.1):

- Always switch off the device and disconnect the charger before cleaning.
- Do not clean the surfaces of the TWIST and its accessories with solvents, alcohol or any other chemicals.
- Do not use abrasive substances, aggressive detergents and high-pressure cleaners.
- Dry the device thoroughly in case it has gotten wet.
- If the device is dirty, soften the dirt as soon as possible and remove it; then dry the device thoroughly.
- Carry out external cleaning of the entire device at least once a month and after every outing in dirty conditions.
- Take care not to get electrical parts (battery, motor, etc.) wet.
- Never immerse the TWIST or its accessories in water.
- Do not wash the TWIST or its accessories with a stream of water of any sort or intensity.
- Dry the parts immediately with a soft cloth after washing them.
- If electrical parts are wet, let them dry and do not use the device until they are dry.

- Use low-pressure compressed air (max. 2 bar) to remove dust and small debris. It is recommended to not insist too much on the front and rear light lens.
- Once dust and small debris have been removed, clean with a wet, soft, non-abrasive cloth.
- Check, in particular, that there are no stones or dirt in the battery compartment that would prevent it from being inserted.
- Check for (and remove) the eventual dirt that could get in under the steering.
- Check for (and remove) the eventual dirt that might have ended up in the Multi-Link Klaxon®.

The TWIST device and accessories are electrically powered. There is a risk of electric shock if they are not cleaned with due care.

20.1 DISINFECTION

Disinfecting the TWIST device is as important as the regular cleaning operation described in this chapter. In order to do so please proceed by using a detergent that complies with the disinfection procedures that respects the materials and components of the device. If in doubt, please refer to the instructions on the packaging of the chosen disinfectant. The disinfection must be done with certified, tested and of proven effectiveness disinfection products that are complying with your national hygiene standards. A list of the currently approved and certified disinfectants can be found on the websites of renowned institutions such as, for example, The European Centre for Disease Prevention and Control (www.ecdc.europa.eu), Centers for Disease Control and Prevention (www.cdc.gov), Robert Koch Institute's website (www.rki.de) and similar.

21. FINGER TRAPS AND SQUEEZING POINTS

The TWIST device and the Linking Systems present some places where fingers could be inadvertently trapped or squeezed during use. Please note and be aware of these points to avoid injury. Below are the various points highlighted in red:

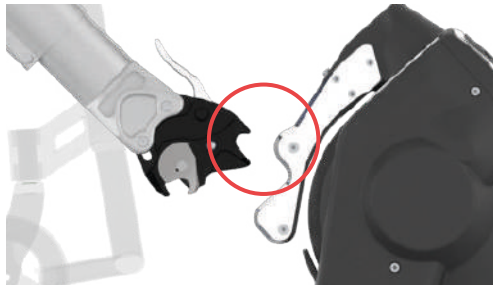
Figure 117

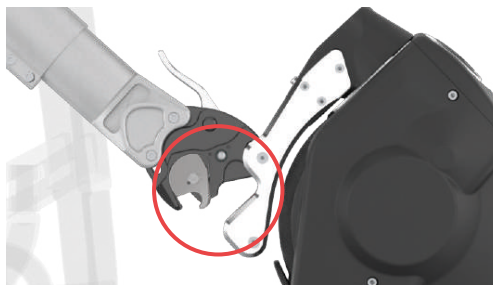
Figure 118

Figure 119

	<p>The squeezing points/finger traps are indicated on the TWIST device by stickers as the one shown here. The stickers are an additional safety measure, in case of doubt please refer to this manual.</p>
	<p>The removal of the stickers indicating the squeezing points is prohibited.</p>

22. DISPOSAL

	<p>The disposal of the equipment inside the TWIST device, the Linking Systems and accessories must absolutely comply with the specific regulations of the country in which they are going to be disposed of. KLAXON is not responsible for disposal that does not comply with the specific regulations in force.</p>
	<p>Recycling of Li-Ion batteries depend from national regulations, ask your Klaxon Dealer for your country rules.</p>

23. PRODUCT WARRANTY

The warranty period is 24 months for the traction device and for the linking systems and 12 months for the batteries, starting from the delivery date of the device.

Natural wear and tear of parts is not covered by the warranty, except for improper wear due to a manufacturing defect.

If a defect occurs during the warranty period, KLAXON will, at its discretion, repair or replace the defective component.

The manufacturer assumes no liability for damages caused by negligence, careless use and maintenance, tampering or incorrect maintenance performed by unauthorised persons, or for failure to follow the instructions for use set out in this Manual.

24. FREE WHEEL/PUSH OPTION

The TWIST wheel has a brushless motor that can rotate freely when turned off. Manual operation and pushing are both possible, without any problems, if it is not possible to use the motor as a propulsion. When installed to a wheelchair, the unit consisting of wheelchair and the TWIST traction device can therefore also be pushed manually without being subject to additional impediment. The mechanical braking system of the handlebar, when installed, allows safe braking even when the unit is switched off or inoperative. Alternatively, the TWIST traction device can also be detached from the wheelchair.

25. PROPERTY TRANSFER

If the TWIST device has been provided by a medical insurance or national health system and is not needed anymore, please contact the involved insurance company, local health system or your local KLAXON Dealer. This action will lead to the assignment of the TWIST device to a new owner.

Everytime the TWIST device changes owner, even in the case of transfer of property through private negotiations, it must undergo maintenance and disinfection prior its first use. Please contact an authorized KLAXON Dealer to perform the maintenance. Please follow the correct cleaning and disinfection procedures as explained on the chapter 20 of the present manual.

26. ORIGINAL TEXT

This document is a translation of the original version written in Italian. Please note that in case of discrepancies between the two versions, the Italian version will be the one considered as official and valid. The Italian version of this manual is available at www.klaxon-klick.com.

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KLAXON MOBILITY GmbH